

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co.S.O.
DATE 1-9-08 TIME 1237
INTOXILYZER SERIAL # 80-001019 LOCATION YESO-PRESCOTT JAIL

- DR 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 2100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- DR 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- DR 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- DR 4. Air blank completed.
- DR 5. Calibration check completed. Test results 0. 098 AC.
- DR 6. Air blank completed.
- DR 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Manuel Q Rains #5-31 / OAS #33026

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

01/09/2008

13:41:13

Standard Lot#: 633901I

Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	13:41:40
Cal Check	0.098	13:41:59
Air Blank	0.000	13:42:28

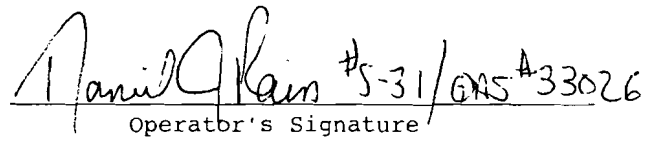

Operator's Signature #5-31/GAS #33026

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISE AGENCY YAVAPAI Co. S.O.
DATE 1-9-07 TIME 1237
INTOXILYZER SERIAL # 80-001019 LOCATION YCSO-PRESCOTT JAIL

OR 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- OR 1. Clock time check.
OR 2. Date check.

OPERATIONAL TESTS

- OR 1. Alcohol-free subject test result 0. 000 AC.
OR 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
OR 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
OR 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

REPEATED PB TEST - OAS ERROR. ADDED 6 MINUTES TO CLOCK.

SIGNATURE Amiel Q. Rain #5-31 / OAS #33026

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number : 80-001019
01/09/2008 13:33:01

Standard Lot#: 633901I
Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #S-31
YAVAPAI CSO

Subject: SQAP,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:33:50
Diagnostic Test	Pass	13:34:17
Air Blank	0.000	13:34:44
0.100 Cal Check	0.098	13:35:03
Air Blank	0.000	13:35:33
Subject Test	0.120*	13:38:41
Air Blank	0.000	13:39:13
Air Blank	0.000	13:39:40
Subject Test	XXX**	13:40:07
Air Blank	0.000	13:40:40

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number : 80-001019
01/09/2008 13:31:58

Standard Lot#: 633901I
Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #S-31
YAVAPAI CSO

Subject: SQAP, TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:32:29
Air Blank	0.000	13:32:57

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number : 80-001019

01/09/2008

13:30:30

Standard Lot#: 633901I

Last Changed By: D. RAISS

#3693

QAS: D. RAISS

#3693

YAVAPAI CSO

Operator: D. RAISS

#S-31

YAVAPAI CSO

Subject: SQAP ,TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	13:31:19
Air Blank	0.000	13:31:48

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number : 80-001019

01/09/2008

13:29:28

Standard Lot#: 633901I

Last Changed By: D. RAISS

#3693

QAS: D. RAISS

#3693

YAVAPAI CSO

Operator: D. RAISS

#S-31

YAVAPAI CSO

Subject: SQAP,TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	ABT*	13:29:57
Air Blank	0.000	13:30:25

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number : 80-001019

01/09/2008

12:37:37

Standard Lot#: 633901I

Last Changed By: D. RAISS

#3693

QAS: D. RAISS

#3693

YAVAPAI CSO

Operator: D. RAISS

#S-31

YAVAPAI CSO

Subject: SQAP,TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	12:39:04
Diagnostic Test	Pass	12:39:32
Air Blank	0.000	12:39:59
0.100 Cal Check	0.098	12:40:18
Air Blank	0.000	12:40:47
Subject Test	0.000	12:41:07
Air Blank	0.000	12:41:34
Five Minute Wait	ABT*	
Air Blank	0.000	12:42:14

*Sequence Aborted

Not a Successfully Completed Test Sequence