

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.

DATE 8-21-08 TIME 0550

INTOXILYZER SERIAL # 80-001019 LOCATION YCSO-PRESCOTT JAIL

- (OK) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (OK) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (OK) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (OK) 4. Air blank completed.
- (OK) 5. Calibration check completed. Test results 0. 099 AC.
- (OK) 6. Air blank completed.
- (OK) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Manuel Q. Rain #5-31/QAS #33026

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

08/21/2008

05:51:36

Standard Lot#: 804301I

Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	05:52:04
Cal Check	0.099	05:52:22
Air Blank	0.000	05:52:52

Daniel J. Reims #5-31 / QAS #33020

Operator's Signature

NEW DRY GAS
EXPIRES 2-14-2010
LOT No. 804301I