

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Daniel McGinnis AGENCY YCSO
DATE 12-12-08 TIME 1823
INTOXILYZER SERIAL # 80-001019 LOCATION PRESCOTT TAIL

- DM* 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .10 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- DM* 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- DM* 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- DM* 4. Air blank completed.
- DM* 5. Calibration check completed. Test results 0. 097 AC.
- DM* 6. Air blank completed.
- DM* 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Daniel McGinnis* 543/04533426

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number : 80-001019
12/12/2008

18:28:57

Standard Lot#: 804301I
Last Changed By: D. RAISS #3693
QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 09/09/1963
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:30:17
Diagnostic Test	Pass	18:30:44
Air Blank	0.000	18:31:12
0.100 Cal Check	0.000	18:31:31
Air Blank	0.000	18:32:00
Subject Test	0.000	18:32:24
Air Blank	ABT*	18:32:33
Air Blank	RFI**	18:32:40

*Sequence Aborted
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number : 80-001019
12/12/2008 18:34:16

Standard Lot#: 804301I
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 09/09/1963
Sex: M Weight: 222

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:35:24
Diagnostic Test	Pass	18:35:51
Air Blank	0.000	18:36:19
0.100 Cal Check	0.098	18:36:38
Air Blank	0.000	18:37:07
Subject Test	0.045*	18:40:15
Air Blank	0.000	18:40:47
Air Blank	0.000	18:41:15
Subject Test	XXX**	18:41:41
Air Blank	IPS***	18:41:48

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***Improper Sample

Not a Successfully Completed Test Sequence

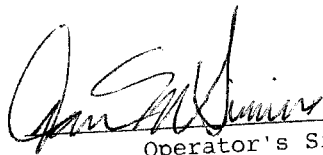
INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number: 80-001019
12/12/2008 18:43:30

Standard Lot#: 804301I
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Test	g/210L	Time
Air Blank	AMB*	18:43:58
Air Blank	0.000	18:44:27

*Ambient Fail


Operator's Signature 8086/QAS #33436

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

12/12/2008

18:45:07

Standard Lot#: 804301I

Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:45:35
Cal Check	0.097	18:45:53
Air Blank	0.000	18:46:23

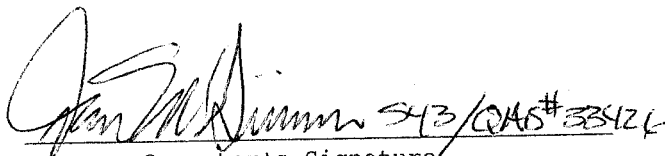

Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. McGinnis AGENCY YLSO
DATE 12-12-08 TIME 1828
INTOXILYZER SERIAL # 80-001019 LOCATION PRESOTT JAIL

DM 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- DM 1. Clock time check.
- DM 2. Date check.

OPERATIONAL TESTS

- DM 1. Alcohol-free subject test result 0. 00 AC.
- DM 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- DM 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- DM 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES DM NO _____

COMMENTS ALL WORKING

SIGNATURE D. McGinnis 8086/QAS # 33420