

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Daniel McGinnis AGENCY YCSO
DATE 2-10-09 TIME 2022
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott Jail

DM 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- DM 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
DM 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
DM 4. Air blank completed.
DM 5. Calibration check completed. Test results 0. 0.018 AC.
DM 6. Air blank completed.
DM 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel McGinnis 8006/QAS#33426

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

02/10/2009

20:22:49

Standard Lot#: 804301I

Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	20:23:16
Cal Check	0.098	20:23:35
Air Blank	0.000	20:24:05

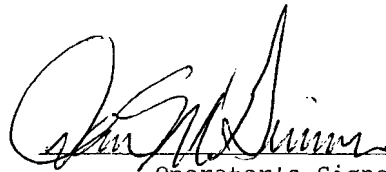
 8086 / QAS # 33426
Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Daniel McGinnis AGENCY YCSO
DATE 02-10-09 TIME 2036
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott Jail

DM 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- DM 1. Clock time check.
DM 2. Date check.

OPERATIONAL TESTS

- DM 1. Alcohol-free subject test result 0. 00 AC.
DM 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
DM 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
DM 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES DM NO

COMMENTS

ALL WORKS

SIGNATURE Daniel McGinnis 8086/QAS #33926

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number : 80-001019
02/10/2009 20:28:26

Standard Lot#: 804301I
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 09/09/1963
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:29:54
Diagnostic Test	Pass	20:30:22
Air Blank	0.000	20:30:49
0.100 Cal Check	0.099	20:31:08
Air Blank	0.000	20:31:37
Subject Test	0.000	20:32:06
Air Blank	0.000	20:32:33
Five Minute Wait	ABT*	
Air Blank	RFI**	20:35:25

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number : 80-001019
02/10/2009 20:36:17

Standard Lot#: 804301I
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 09/09/1963
Sex: M Weight: 190

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:37:19
Diagnostic Test	Pass	20:37:46
Air Blank	0.000	20:38:14
0.100 Cal Check	0.098	20:38:33
Air Blank	0.000	20:39:03
Subject Test	0.026*	20:42:11
Air Blank	IPS**	20:42:19
Air Blank	0.000	20:42:47

*Deficient Sample

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number : 80-001019
02/10/2009 20:43:30

Standard Lot#: 804301I
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 09/09/1963
Sex: M Weight: 190

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:44:43
Diagnostic Test	Pass	20:45:10
Air Blank	0.000	20:45:37
0.100 Cal Check	0.097	20:45:56
Air Blank	0.000	20:46:26
Subject Test	XXX*	20:46:53
Air Blank	0.000	20:47:27

*Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Daniel McGinnis AGENCY 1 VCSO
DATE 02-10-09 TIME 2048
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott Jail

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution, 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean-dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 097 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel McGinnis 8086/QAS# 33426

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

02/10/2009

20:48:04


Standard Lot#: 804301I

Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	20:48:31
Cal Check	0.097	20:48:50
Air Blank	0.000	20:49:20

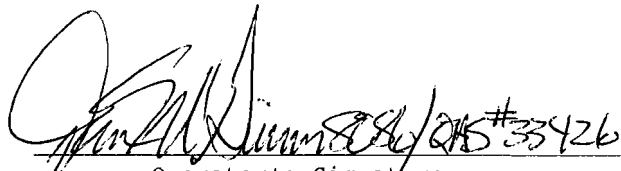
Operator's Signature