

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Daniel McGinnis AGENCY YCSO

DATE 3-25-10 TIME 2056

INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

DM

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.10 AC.
OR

~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~

DM
DM
DM
DM
DM

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 0.18 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Daniel McGinnis* 543/QAG # 33426

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

03/25/2010

20:56:01

Standard Lot#: 712322

Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Operator: D. MCGINNIS #8086

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	20:56:33
0.100 Cal Check	0.098	20:56:53
Air Blank	0.000	20:57:23

Preventative Maintenance
Performed -- Timer Reset

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

03/25/2010

20:50:47

Standard Lot#: 712322

Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693

YAVAPAI CSO

Operator: D. MCGINNIS #8086

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 20:51:24
0.100 Cal Check 0.098 20:51:45
Air Blank 0.000 20:52:14
Air Blank 0.000 20:54:43

*Input Timed Out

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Daniel McGinnis AGENCY YCSO
DATE 3-25-10 TIME 2103
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

DM 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- DM 1. Clock time check.
DM 2. Date check.

OPERATIONAL TESTS

- DM 1. Alcohol-free subject test result 0. 00 AC.
DM 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
DM 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
DM 4. Standard Calibration Check standard 0. 10 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES DM NO _____

COMMENTS RF1 failed to fault, I was unfamiliar with software update, had to consult Sgt Raiss. RF1 did fault and was functioning properly

SIGNATURE Daniel McGinnis 543/QAS # 33426

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number: 80-001019
Core Version: 8105.48
03/25/2010 20:57:48

Standard Lot#: 712322
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 09/09/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:58:50
Diagnostic Test	Pass	20:59:25
Air Blank	0.000	20:59:52
0.100 Cal Check	0.098	21:00:13
Air Blank	0.000	21:00:43
Subject Test	0.000	21:01:03
Air Blank	ABT*	21:01:10
Air Blank	0.000	21:01:39

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

03/25/2010

21:02:06

Standard Lot#: 712322

Last Changed By: D. RAISS

#3693

QAS: D. MCGINNIS

#8086

YAVAPAI CSO

Operator: D. MCGINNIS

#8086

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 09/09/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:03:07
Diagnostic Test	Fail*	21:03:17
Air Blank	0.000	21:03:46

* Analytical Stability Test

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT JAIL
Serial Number: 80-001019
Core Version: 8105.48
03/25/2010 21:03:52

Standard Lot#: 712322
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 09/09/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:04:59
Diagnostic Test	Pass	21:05:34
Air Blank	0.000	21:06:02
0.100 Cal Check	0.097	21:06:23
Air Blank	0.000	21:06:53
Subject Test	0.077*	21:10:00
Air Blank	0.000	21:10:33
Air Blank	0.000	21:11:00
Subject Test	XXX**	21:11:28
Air Blank	IPS***	21:11:36

*Deficient Sample
**Invalid Sample
 Begin new deprivation period
***Improper Sample

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Daniel McGinnis AGENCY YCSO
DATE 3-25-10 TIME 2112
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- DM* 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a ~~standard alcohol concentration solution~~ AC, into a ~~clean dry~~ simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach ~~34°C ± 0.2°C~~
- DM* 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- DM* 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- DM* 4. Air blank completed.
- DM* 5. Calibration check completed. Test results 0. .097 AC.
- DM* 6. Air blank completed.
- DM* 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Daniel McGinnis 543/QAS #33426

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

03/25/2010

21:12:21

Standard Lot#: 712322

Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Operator: D. MCGINNIS #8086

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	21:12:52
0.100 Cal Check	0.097	21:13:13
Air Blank	0.000	21:13:43

Preventative Maintenance
Performed -- Timer Reset