

**COPY**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray AGENCY YCSO  
DATE 5/29/10 TIME 7 29 24  
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott YCSO

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray

DPS Form Exh G-2 (Rev 05-01)  
Tank # 712322

TNTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

05/29/2010

07:29:24

-----  
Standard Lot#: 712322

Last Changed By: D. RAISS #3693

-----  
QAS: J. MURRAY #5597

YAVAPAI CSO

-----  
Operator: J. MURRAY #5597

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	07:29:59
0.100 Cal Check	0.098	07:30:19
Air Blank	0.000	07:30:49

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST M. J. Murray AGENCY YCSO  
DATE 5/29/10 TIME 12:23:36  
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott YCSO

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 150 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

Bottle # 712322 Approx 800  
Ambient Fail x2 - Instrument powered down, turned on.  
Instrument QASed & Standard checked O.K. (JIM)

SIGNATURE J. Murray

INTOXILYZER 8000  
Location: YCSO PRESCOTT JAIL  
Serial Number: 80-001019  
Core Version: 8105.48  
05/29/2010 12:10:25

---

Standard Lot#: 712322  
Last Changed By: D. RAISS #3693

---

QAS: J. MURRAY #5597  
YAVAPAI CSO

---

Operator: J. MURRAY #5597  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:11:19
Diagnostic Test	Pass	12:11:54
Air Blank	0.000	12:12:22
0.100 Cal Check	0.098	12:12:43
Air Blank	AMB*	12:13:13
Air Blank	AMB*	12:13:42

\*Ambient Fail

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO PRESCOTT JAIL  
Serial Number: 80-001019  
Core Version: 8105.48  
05/29/2010 12:03:56

---

Standard Lot#: 712322  
Last Changed By: D. RAISS #3693

---

QAS: J. MURRAY #5597  
YAVAPAI CSO

---

Operator: J. MURRAY #5597  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:05:23
Diagnostic Test	Pass	12:05:58
Air Blank	0.000	12:06:26
0.100 Cal Check	0.097	12:06:47
Air Blank	0.000	12:07:16
Subject Test	0.000	12:07:37
Air Blank	0.000	12:08:05
Wait	ABT*	
Air Blank	AMB**	12:10:03

\*Sequence Aborted

\*\*Ambient Fail

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

05/29/2010

12:23:36

---

Standard Lot#: 712322

Last Changed By: D. RAISS

#3693

---

QAS: J. MURRAY

#5597

YAVAPAI CSO

---

Operator: J. MURRAY

#5597

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	12:24:26
Air Blank	IPS**	12:24:44

\*RFI Detect

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

05/29/2010

12:24:54

---

Standard Lot#: 712322

Last Changed By: D. RAISS

#3693

---

QAS: J. MURRAY

#5597

YAVAPAI CSO

---

Operator: J. MURRAY

#5597

YAVAPAI CSO

---

Subject: SOAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:25:44
Diagnostic Test	Pass	12:26:19
Air Blank	0.000	12:26:47
0.100 Cal Check	0.097	12:27:08
Air Blank	0.000	12:27:37
Subject Test	0.000	12:27:51
Air Blank	ABT*	12:28:01
Air Blank	IPS**	12:28:12

\*Sequence Aborted

\*\*Improper Sample

---

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YCSO PRESCOTT JAIL

Serial Number: 80-001019

Core Version: 8105.48

05/29/2010

12:29:04

---

Standard Lot#: 712322

Last Changed By: D. RAISS

#3693

---

QAS: J. MURRAY

#5597

YAVAPAI CSO

---

Operator: J. MURRAY

#5597

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:29:54
Diagnostic Test	Pass	12:30:29
Air Blank	0.000	12:30:56
0.100 Cal Check	0.098	12:31:17
Air Blank	0.000	12:31:46
Subject Test	0.100*	12:34:53
Air Blank	0.000	12:35:25
Air Blank	0.000	12:35:53
Subject Test	XXX**	12:36:26
Air Blank	0.000	12:36:59

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. M. V. S. J. AGENCY Y250  
DATE 5/29/10 TIME 1237 38  
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott Y250

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution            AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 095 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. M. V. S. J.

INTOXILYZER 8000  
Location: YCSO PRESCOTT JAIL  
Serial Number: 80-001019  
Core Version: 8105.48  
05/29/2010 12:37:38

-----  
Standard Lot#: 712322  
Last Changed By: D. RAISS #3693  
-----

QAS: J. MURRAY #5597  
YAVAPAI CSO  
-----

Operator: J. MURRAY #5597  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	12:38:09
0.100 Cal Check	0.095	12:38:30
Air Blank	0.000	12:39:00

Preventative Maintenance  
Performed -- Timer Reset