

COPY

**EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)**

**ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000**

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray NSB AGENCY Yavapai County Sheriff
DATE 2/6/11 TIME 06:51:41
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott Office

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 096 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NSB

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

02/06/2011

06:51:41

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56

YAVAPAI YCSO

Operator: J. MURRAY #N56

YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	06:52:28
0.100 Cal Check	0.096	06:52:49
Air Blank	0.000	06:53:18

Preventative Maintenance
Performed -- Timer Reset

COPY

**EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)**

**ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000**

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Murray NSB AGENCY Yavapai County Sheriff's
DATE 2/6/11 TIME 07:31:13
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott office

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

25310100A1 - exp 10/1/12

SIGNATURE J. Murray NSB

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

02/06/2011

07:31:13

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:32:40
Diagnostic Test	Pass	07:33:15
Air Blank	0.000	07:33:43
0.100 Cal Check	0.096	07:34:03
Air Blank	0.000	07:34:33
Subject Test	0.000	07:34:52
Air Blank	0.000	07:35:20
Wait	ABT*	
Air Blank	RFI**	07:35:45

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

02/06/2011

07:35:54

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	07:36:46
Air Blank	RFI**	07:36:56

*Improper Sample

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

02/06/2011

07:37:07

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:38:16
Diagnostic Test	Pass	07:38:51
Air Blank	0.000	07:39:19
0.100 Cal Check	0.095	07:39:40
Air Blank	0.000	07:40:09
Subject Test	0.103*	07:43:16
Air Blank	0.000	07:43:48
Air Blank	0.000	07:44:15
Subject Test	XXX**	07:44:49
Air Blank	0.000	07:45:21

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

COPY

**EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)**

**ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000**

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray NSB AGENCY Yavapai County Sheriff's
DATE 2/6/11 TIME 07:46:14
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott 103 Intox

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (7) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(7) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(4) 4. Air blank completed.
(X) 5. Calibration check completed. Test results 0. 095 AC.
(7) 6. Air blank completed.
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NSB

INTOXILYZER 8000
Location: YCSO PRESCOTT PATROL
Serial Number: 80-001019
Core Version: 8105.48
02/06/2011 07:46:14

Standard Lot#: 25310100A1
Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56
YAVAPAI YCSO

Operator: J. MURRAY #N56
YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	07:46:51
0.100 Cal Check	0.095	07:47:12
Air Blank	0.000	07:47:42

Preventative Maintenance
Performed -- Timer Reset