

**COPY**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office  
DATE 2/27/11 TIME 15:17:11  
INTOXILYZER SERIAL # 80-001019 LOCATION Piscott Intox

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.095 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray MSB

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

02/27/2011

15:16:43

-----  
Standard Lot#: 25310100A1

Last Changed By: J. MURRAY #N56  
-----

QAS: J. MURRAY #N56

YAVAPAI YCSO  
-----

Operator: J. MURRAY #N56

YAVAPAI YCSO  
-----

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 15:17:20 |
| 0.100 Cal Check | 0.095  | 15:17:40 |
| Air Blank       | 0.000  | 15:18:10 |

Preventative Maintenance  
Performed -- Timer Reset

**COPY**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai Co. Sheriff's  
DATE 2/27/11 TIME 15:18:56  
INTOXILYZER SERIAL # 80-001019 LOCATION Piuslett Intox

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.095 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NSB

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

02/27/2011

15:13:17

-----  
Standard Lot#: 25310100A1

Last Changed By: J. MURRAY #N56  
-----

QAS: J. MURRAY #N56

YAVAPAI YCSO  
-----

Operator: J. MURRAY #N56

YAVAPAI YCSO  
-----

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 15:13:56 |
| 0.100 Cal Check | 0.095  | 15:14:17 |
| Air Blank       | 0.000  | 15:14:47 |

Preventative Maintenance  
Performed -- Timer Reset