

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. MURRAY NSB AGENCY Yavapai County Sheriff's Office
DATE 3/8/11 TIME 06:49:28
INTOXILYZER SERIAL # 80-001019 LOCATION Piscott Intox Room

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.095 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. MURRAY NSB

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

03/08/2011

06:49:28

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56

YAVAPAI YCSO

Operator: J. MURRAY #N56

YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	06:50:05
0.100 Cal Check	0.095	06:50:25
Air Blank	0.000	06:50:55

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office
DATE 3/8/11 TIME 06 54 48
INTOXILYZER SERIAL # 80-001019 LOCATION Piscott In'ox Room

(x) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (x) 1. Clock time check.
- (x) 2. Date check.

OPERATIONAL TESTS

- (x) 1. Alcohol-free subject test result 0. 000 AC.
- (x) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (x) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (x) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS Serial # 25310100A1 .100 standard
+900 PSI

SIGNATURE J. Murray NSB

INTOXILYZER 8000
Location: YCSO PRESCOTT PATROL
Serial Number: 80-001019
Core Version: 8105.48
03/08/2011 06:54:48

Standard Lot#: 25310100A1
Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56
YAVAPAI YCSO

Operator: J. MURRAY #N56
YAVAPAI YCSO

Subject: SQAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	06:55:52
Diagnostic Test	Pass	06:56:26
Air Blank	0.000	06:56:53
0.100 Cal Check	0.095	06:57:14
Air Blank	0.000	06:57:43
Subject Test	0.000	06:58:10
Air Blank	0.000	06:58:37
Wait	ABT*	
Air Blank	IPS**	06:59:21

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT PATROL
Serial Number: 80-001019
Core Version: 8105.48
03/08/2011 07:08:46

Standard Lot#: 25310100A1
Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56
YAVAPAI YCSO

Operator: J. MURRAY #N56
YAVAPAI YCSO

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	07:09:58
Air Blank	IPS**	07:10:06

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT PATROL
Serial Number: 80-001019
Core Version: 8105.48
03/08/2011 07:10:54

Standard Lot#: 25310100A1
Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56
YAVAPAI YCSO

Operator: J. MURRAY #N56
YAVAPAI YCSO

Subject: SQAP, TESH
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:12:04
Diagnostic Test	Pass	07:12:39
Air Blank	0.000	07:13:06
0.100 Cal Check	0.095	07:13:27
Air Blank	0.000	07:13:56
Subject Test	0.068*	07:17:04
Air Blank	0.000	07:17:35
Air Blank	0.000	07:18:02
Subject Test	XXX**	07:18:26
Air Blank	0.000	07:18:58

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray N56 AGENCY Yavapai County Sheriff's Office
DATE 3/8/11 TIME 07 19 37
INTOXILYZER SERIAL # 80-001019 LOCATION Piscott Intox Room

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✗) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✗) 4. Air blank completed.
- (7) 5. Calibration check completed. Test results 0. 095 AC.
- (✗) 6. Air blank completed.
- (✗) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray N56

INTOXILYZER 8000

Location: YCSO PRESCOTT PATROL

Serial Number: 80-001019

Core Version: 8105.48

03/08/2011

07:19:37

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56

YAVAPAI YCSO

Operator: J. MURRAY #N56

YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	07:20:13
0.100 Cal Check	0.095	07:20:34
Air Blank	0.000	07:21:03

Preventative Maintenance
Performed -- Timer Reset