

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST JEFFREY R. JAEGER AGENCY YAVAPAI COUNTY SHERIFF'S OFFICE
DATE 03-16-2012 TIME 1628
INTOXILYZER SERIAL # 80-001019 LOCATION DUI TRAILER - (YCSO)

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- * 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
 4. Air blank completed.
 5. Calibration check completed. Test results 0.094 AC.
 6. Air blank completed.
 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  #3566

DPS Form Exh G-2 (Rev 05-01)

* DISPLAY = "CONTACT QAS 31 DAY PREV MAINT"

INTOXILYZER 8000
Location: YCSO DUI TRAILER
Serial Number: 80-001019
Core Version: 8105.48
03/16/2012

16:28:33

Standard Lot#: 25310100A1
Last Changed By: J. MURRAY #N56

QAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	16:29:23
0.100 Cal Check	0.094	16:29:44
Air Blank	0.000	16:30:14

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST JEFFREY R. JAEGER AGENCY YAVAPAI COUNTY SHERIFF'S OFFICE

DATE 03-16-2012 TIME 1637

INTOXILYZER SERIAL # 80-001019 LOCATION DUI TRAILER - (YCSO)

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0.000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0.100 AC. Result 0.093 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

INSTRUMENT IS OPERATING PROPERLY AND ACCURATELY

SIGNATURE  #3566

INTOXILYZER 8000
Location: YCSO DUI TRAILER
Serial Number: 80-001019
Core Version: 8105.48
03/16/2012

16:37:00

Standard Lot#: 25310100A1
Last Changed By: J. MURRAY #N56

QAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 06/28/1985
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:38:50
Diagnostic Test	Pass	16:39:25
Air Blank	0.000	16:39:52
0.100 Cal Check	0.094	16:40:13
Air Blank	0.000	16:40:43
Subject Test	0.000	16:41:05
Air Blank	0.000	16:41:32
Wait	ABT*	
Air Blank	RFI**	16:42:02

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO DUI TRAILER

Serial Number: 80-001019

Core Version: 8105.48

03/16/2012

16:43:46

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY

#N56

QAS: J. JAEGER

#3566

YAVAPAI CSO

Operator: J. JAEGER

#3566

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 06/28/1985

Sex: M

Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:44:59
Diagnostic Test	Pass	16:45:34
Air Blank	0.000	16:46:01
0.100 Cal Check	0.093	16:46:22
Air Blank	0.000	16:46:52
Subject Test	0.055*	16:49:59
Air Blank	0.000	16:50:31
Air Blank	0.000	16:50:58
Subject Test	XXX**	16:51:26
Air Blank	IPS***	16:51:37

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***Improper Sample

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

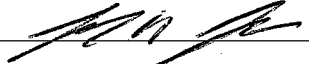
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST JEFFREY R. JAEGER AGENCY YAVAPAI COUNTY SHERIFF'S OFFICE

DATE 03-16-2012 TIME 1657

INTOXILYZER SERIAL # 80-001019 LOCATION DUI TRAILER - (YCSO)

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.093 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  #3566

INTOXILYZER 8000

Location: YCSO DUI TRAILER

Serial Number: 80-001019

Core Version: 8105.48

03/16/2012

16:57:54

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY #N56

QAS: J. JAEGER #3566

YAVAPAI CSO

Operator: J. JAEGER #3566

YAVAPAI CSO

Test	g/210L	Time
Air Blank	AMB*	16:58:26
Air Blank	0.000	16:58:54

*Ambient Fail

INTOXILYZER 8000

Location: YCSO DUI TRAILER

Serial Number: 80-001019

Core Version: 8105.48

03/16/2012

16:59:04

Standard Lot#: 25310100A1

Last Changed By: J. MURRAY #N56

QAS: J. JAEGER #3566

YAVAPAI CSO

Operator: J. JAEGER #3566

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	16:59:35
0.100 Cal Check	0.093	16:59:56
Air Blank	0.000	17:00:25

Preventative Maintenance
Performed -- Timer Reset