

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray NS6/5597 AGENCY YCSO
DATE 6/19/13 TIME 10:12:01
INTOXILYZER SERIAL # 80-001019 LOCATION Picasset 103

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 094 AC.
(✓) 6. Air blank completed.
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS6/5597

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

06/19/2013

10:12:01

Standard Lot#: 32012100A1

Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56

YAVAPAI YCSO

Operator: J. MURRAY #N56

YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	10:12:38
0.100 Cal Check	0.094	10:12:59
Air Blank	0.000	10:13:30

Preventative Maintenance
Performed -- Timer Reset