

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murray AGENCY Yavapai County Sheriff's Office
DATE 8/31/14 TIME 9:28:07
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- (~~2~~) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution — AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (~~2~~) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (~~2~~) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (~~2~~) 4. Air blank completed.
- (~~2~~) 5. Calibration check completed. Test results 0. 096 AC.
- (~~2~~) 6. Air blank completed.
- (~~2~~) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS6/5597

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/31/2014 09:28:07

Standard Lot#: 15013100A2
Last Changed By: J. SLOWINSKI #6101

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	09:28:50
0.100 Cal Check	0.096	09:29:10
Air Blank	0.000	09:29:40

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office
DATE 8/31/14 TIME 09:30:49
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

15013100A2 450 PSI
6/1/2015

SIGNATURE J. Murray NS2/5597

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/31/2014

09:30:49

Standard Lot#: 15013100A2
Last Changed By: J. SLOWINSKI #6101

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:31:54
Diagnostic Test	Pass	09:32:29
Air Blank	0.000	09:32:56
0.100 Cal Check	0.096	09:33:17
Air Blank	0.000	09:33:46
Subject Test	0.000	09:34:40
Air Blank	0.000	09:35:08
Wait	ABT*	
Air Blank	RFI**	09:35:28

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/31/2014 09:35:37

Standard Lot#: 15013100A2
Last Changed By: J. SLOWINSKI #6101

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	09:36:26
Air Blank	RFI**	09:36:35

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/31/2014 09:37:02

Standard Lot#: 15013100A2
Last Changed By: J. SLOWINSKI #6101

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:38:13
Diagnostic Test	Pass	09:38:47
Air Blank	0.000	09:39:15
0.100 Cal Check	0.096	09:39:36
Air Blank	0.000	09:40:05
Subject Test	0.095*	09:43:13
Air Blank	0.000	09:43:44
Air Blank	0.000	09:44:12
Subject Test	XXX**	09:44:50
Air Blank	0.000	09:45:23

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Musial AGENCY Yavapai County Sheriff's Office
DATE 8/31/17 TIME 09:45:49
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.096 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Musial NSB/5597

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/31/2014 09:45:49

Standard Lot#: 15013100A2
Last Changed By: J. SLOWINSKI #6101

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	09:46:29
0.100 Cal Check	0.096	09:46:50
Air Blank	0.000	09:47:20

Preventative Maintenance
Performed -- Timer Reset