

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murray AGENCY Yavapai County Sheriff's Office
DATE 3/30/15 TIME 07:58:34
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0. 098 AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
03/30/2015 07:58:34

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	07:59:16
0.100 Cal Check	0.098	07:59:37
Air Blank	0.000	08:00:06

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Murray AGENCY Yavapai County Sheriff's Office
DATE 3/30/15 TIME 08:02:15
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
- (X) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0. 000 AC.
- (X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES X NO

COMMENTS 25114/00A4 750 PSI
10/5/2016

SIGNATURE J. Murray

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
03/30/2015 08:02:15

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:03:13
Diagnostic Test	Pass	08:03:48
Air Blank	0.000	08:04:15
0.100 Cal Check	0.097	08:04:36
Air Blank	0.000	08:05:05
Subject Test	0.000	08:06:06
Air Blank	0.000	08:06:34
Wait	ABT*	
Air Blank	IPS**	08:06:49

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
03/30/2015

08:06:53

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:07:31
Air Blank	IPS**	08:07:42

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
03/30/2015 08:08:32

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:09:28
Diagnostic Test	Pass	08:10:03
Air Blank	0.000	08:10:31
0.100 Cal Check	0.097	08:10:51
Air Blank	0.000	08:11:21
Subject Test	XXX*	08:11:53
Air Blank	0.000	08:12:29

*Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
03/30/2015

08:12:33

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:13:28
Diagnostic Test	Pass	08:14:03
Air Blank	0.000	08:14:30
0.100 Cal Check	0.097	08:14:51
Air Blank	0.000	08:15:21
Subject Test	0.148*	08:18:28
Air Blank	0.000	08:19:00
Air Blank	0.000	08:19:27
Subject Test	XXX**	08:19:53
Air Blank	0.000	08:20:26

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murray AGENCY Yavapai County Sheriff's Office
DATE 3/30/15 TIME 08:20:53
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 096 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray :

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

03/30/2015

08:20:53

Standard Lot#: 25414100A4

Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	08:21:30
0.100 Cal Check	0.096	08:21:51
Air Blank	0.000	08:22:21

Preventative Maintenance
Performed -- Timer Reset