

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murray AGENCY Yavapai County Sheriff's Office
DATE 8/26/15 TIME 21:30:39
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- (+) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (+) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (+) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (+) 4. Air blank completed.
- (+) 5. Calibration check completed. Test results 0. 096 AC.
- (+) 6. Air blank completed.
- (+) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS2/5597

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/26/2015 21:30:39

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	21:31:16
0.100 Cal Check	0.096	21:31:37
Air Blank	0.000	21:32:06

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Murray AGENCY Yavapai County Sheriff's Office
DATE 8/26/15 TIME 21:33:12
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (+) 1. Clock time check.
(+) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0. 000 AC.
(X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

Gas Bottle ~~25414100A4~~ 300 PSI
exp 10/5/16 25414100A4

SIGNATURE J. Murray NSB/5597

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/26/2015

21:33:12

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:34:11
Diagnostic Test	Pass	21:34:46
Air Blank	0.000	21:35:14
0.100 Cal Check	0.096	21:35:34
Air Blank	0.000	21:36:04
Subject Test	0.000	21:37:28
Air Blank	0.000	21:37:55
Wait	ABT*	
Air Blank	RFI**	21:38:11

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/26/2015 21:38:15

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	21:38:58
Air Blank	0.000	21:39:27

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/26/2015

21:39:31

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP, TESH
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:40:39
Diagnostic Test	Pass	21:41:14
Air Blank	0.000	21:41:41
0.100 Cal Check	0.095	21:42:02
Air Blank	0.000	21:42:32
Subject Test	RGE*	21:43:05
Air Blank	0.000	21:43:39

*Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/26/2015 21:43:46

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:45:01
Diagnostic Test	Pass	21:45:36
Air Blank	0.000	21:46:03
0.100 Cal Check	0.096	21:46:24
Air Blank	0.000	21:46:53
Subject Test	0.189*	21:50:01
Air Blank	0.000	21:50:32
Air Blank	0.000	21:51:00
Subject Test	XXX**	21:51:50
Air Blank	0.000	21:52:25

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murray AGENCY Yuma County Sheriff's Office
DATE 8/26/15 TIME 21:52:45
INTOXILYZER SERIAL # 80-001019 LOCATION Pescott

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 1.10 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0. 095 AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS6/5597

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/26/2015 21:52:45

Standard Lot#: 25414100A4
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	21:53:26
0.100 Cal Check	0.095	21:53:47
Air Blank	0.000	21:54:16

Preventative Maintenance
Performed -- Timer Reset