

COPY

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Blackley AGENCY Maricopa County SO  
DATE 3-27-16 TIME 11:43.53  
INTOXILYZER SERIAL # 80-001019 LOCATION Brown Mesa

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.099 AC.
- (X) 6. Air blank completed.
- ( ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

Tank Pressure 700 psi  
Serial # 3631410044  
Exp 03-05-17

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

03/27/2016

11:40:10

-----  
Standard Lot#: 36314100A4

Last Changed By: S. BLAKLEY SR #L6861

-----  
QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

-----  
Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	11:40:46
0.100 Cal Check	0.099	11:41:07
Air Blank	RFI*	11:41:22
Air Blank	0.000	11:41:50

\*RFI Detect

*Portable Radio is on "Weyrd"*

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

03/27/2016

11:43:53

-----  
Standard Lot#: 36314100A4

Last Changed By: S. BLAKLEY SR #L6861  
-----

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	11:44:26
0.100 Cal Check	0.099	11:44:47
Air Blank	0.000	11:45:17

Preventative Maintenance

Performed -- Timer Reset



EXHIBIT G-2  
 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)  
 ARIZONA DEPARTMENT OF PUBLIC SAFETY  
 STANDARD QUALITY ASSURANCE PROCEDURES  
 INTOXILYZER MODEL 8000  
 STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. B. Kelly AGENCY Yavapai County SO  
 DATE 3-27-16 TIME 12:13:58  
 INTOXILYZER SERIAL # 80-001019 LOCATION Prescott YCSO

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
 OR  
 Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0.098 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

03/27/2016

12:13:58

-----  
Standard Lot#: 36314100A4

Last Changed By: S. BLAKLEY SR #L6861  
-----

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	12:14:29
0.100 Cal Check	0.098	12:14:50
Air Blank	0.000	12:15:19

Preventative Maintenance

Performed -- Timer Reset