

**COPY**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO  
DATE 5-22-16 TIME 07:25:43  
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution — AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.097 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blakley Sr.

DPS Form Exh G-2 (Rev 05-01)

Serial # 36314100 A4  
Exp 2-5-2017  
PSI 650

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

05/22/2016

07:25:05

-----  
Standard Lot#: 36314100A4

Last Changed By: S. BLAKLEY SR #L6861  
-----

QAS: s. blakley sr #16861

yavapai cso  
-----

Operator: s. blakley sr #16861

yavapai cso  
-----

Test	g/210L	Time
Air Blank	0.000	07:25:43
0.100 Cal Check	0.097	07:26:04
Air Blank	0.000	07:26:33

Preventative Maintenance  
Performed -- Timer Reset