

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO

DATE 11-17-16 TIME 14:58:01

INTOXILYZER SERIAL # 80-001019 LOCATION Act

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0.098 AC.
- (x) 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blakley Sr.

Tank Ser. 3631410044
Exp 02-05-2017
APPS 200 PSI

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

11/17/2016

14:58:01

Standard Lot#: 36314100A4

Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	14:58:37
0.100 Cal Check	0.098	14:58:57
Air Blank	0.000	14:59:27

Preventative Maintenance
Performed -- Timer Reset

MO