

COPY

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Scott E Blakley Sr. AGENCY Yavapai County SO  
DATE 6-6-17 TIME 23:43:04  
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
- (X) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0. 000 AC.
- (X) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (X) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES X NO     

COMMENTS

Additional inspection done because instrument would not come  
out of "standby" mode. No problems found.

SIGNATURE Scott E Blakley Sr.

RECEIVED

JUN 12 2017

YAVAPAI COUNTY ATTORNEY

DR17-019797

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

06/06/2017

23:43:04

---

Standard Lot#: 27016100A1

Last Changed By: S. BLAKLEY SR #L6861

---

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	23:44:06
Diagnostic Test	RFI*	23:44:18
Air Blank	0.000	23:44:46

\* RFI Detect

**Not a Successfully Completed Test Sequence**

DR17-019147

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

06/06/2017

23:45:06

---

Standard Lot#: 27016100A1

Last Changed By: S. BLAKLEY SR #L6861

---

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	23:46:04
Diagnostic Test	Pass	23:46:39
Air Blank	0.000	23:47:06
0.100 Cal Check	0.097	23:47:27
Air Blank	0.000	23:47:57
Subject Test	0.000	23:48:14
Air Blank	0.000	23:48:41
Wait	ABT*	
Air Blank	0.000	23:50:06

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO PRESCOTT  
Serial Number: 80-001019  
Core Version: 8105.48  
06/06/2017 23:50:50

---

Standard Lot#: 27016100A1  
Last Changed By: S. BLAKLEY SR #L6861

---

QAS: S. BLAKLEY SR #L6861  
YAVAPAI CSO

---

Operator: S. BLAKLEY SR #L6861  
YAVAPAI CSO

---

Subject: SQAP, TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	23:51:49
Diagnostic Test	Pass	23:52:24
Air Blank	0.000	23:52:51
0.100 Cal Check	0.098	23:53:12
Air Blank	IPS*	23:53:25
Air Blank	0.000	23:53:54

\*Improper Sample

**Not a Successfully Completed Test Sequence**

DK17-019711

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

06/06/2017

23:54:35

---

Standard Lot#: 27016100A1

Last Changed By: S. BLAKLEY SR #L6861

---

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

---

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	23:55:33
Diagnostic Test	Pass	23:56:08
Air Blank	0.000	23:56:36
0.100 Cal Check	0.098	23:56:56
Air Blank	0.000	23:57:26
Subject Test	0.070*	00:00:33
Air Blank	0.000	00:01:05
Air Blank	0.000	00:01:32
Subject Test	XXX**	00:02:13
Air Blank	0.000	00:02:47

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

06/07/2017

00:03:31

-----  
Standard Lot#: 27016100A1

Last Changed By: S. BLAKLEY SR #L6861  
-----

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	00:04:00
0.100 Cal Check	0.098	00:04:21
Air Blank	0.000	00:04:51

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott E Blahley Sr AGENCY Yavapai County SO  
DATE 6-6-17 TIME 23:40:26  
INTOXILYZER SERIAL # 80-001019 LOCATION Prescott

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott E Blahley Sr

Tank Serial # 27016100A1  
Exp 11-5-2018  
Pres 850 psi

DR17-019111

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

06/06/2017

23:40:26

-----  
Standard Lot#: 27016100A1

Last Changed By: S. BLAKLEY SR #L6861

-----  
QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

-----  
Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	23:40:58
0.100 Cal Check	0.098	23:41:19
Air Blank	0.000	23:41:48

Preventative Maintenance  
Performed -- Timer Reset