

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO
DATE 8-23-17 TIME 21:17:03
INTOXILYZER SERIAL # 80-001019 LOCATION YCSO Prescott

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.112 AC.
OR
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0. 098 AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blakley Sr.

Tank Serial # 27016100A1
Exp 11-5-2018
Pres 600 psi

DR 17-030296

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/23/2017 21:17:03

Standard Lot#: 27016100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 21:17:58 |
| 0.100 Cal Check | 0.098 | 21:18:19 |
| Air Blank | 0.000 | 21:18:48 |

Preventative Maintenance
Performed -- Timer Reset

AR 17-030296

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO

DATE 8-23-17 TIME 21:20:16

INTOXILYZER SERIAL # 80-001019 LOCATION YCSO Prescott

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Scott Blakley Sr.

DR 17-030296

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/23/2017 21:20:16

Standard Lot#: 27016100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SQAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 21:21:25 |
| Diagnostic Test | Pass | 21:22:01 |
| Air Blank | 0.000 | 21:22:28 |
| 0.100 Cal Check | 0.098 | 21:22:49 |
| Air Blank | 0.000 | 21:23:18 |
| Subject Test | 0.000 | 21:23:44 |
| Air Blank | 0.000 | 21:24:12 |
| Wait | ABT* | |
| Air Blank | 0.000 | 21:24:48 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

DR 17-030296

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/23/2017 21:25:37

Standard Lot#: 27016100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SQAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | IPS* | 21:26:27 |
| Air Blank | RFI** | 21:26:47 |

*Improper Sample
**RFI Detect

Not a Successfully Completed Test Sequence

DR 17-030296

INTOXILYZER 8000

Location: YCSO PRESCOTT

Serial Number: 80-001019

Core Version: 8105.48

08/23/2017

21:27:34

Standard Lot#: 27016100A1

Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F Weight: 115

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 21:28:44 |
| Diagnostic Test | Pass | 21:29:19 |
| Air Blank | 0.000 | 21:29:46 |
| 0.100 Cal Check | 0.098 | 21:30:07 |
| Air Blank | 0.000 | 21:30:37 |
| Subject Test | 0.164* | 21:33:44 |
| Air Blank | 0.000 | 21:34:16 |
| Air Blank | 0.000 | 21:34:44 |
| Subject Test | XXX** | 21:35:04 |
| Air Blank | 0.000 | 21:35:38 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

DR 17-030296

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/23/2017 21:36:31

Standard Lot#: 27016100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 21:37:02 |
| 0.100 Cal Check | 0.097 | 21:37:22 |
| Air Blank | 0.000 | 21:37:52 |

Preventative Maintenance
Performed -- Timer Reset

DR 17-030296

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO

DATE 8-23-17 TIME _____

INTOXILYZER SERIAL # 80-021019 LOCATION YCSO Prescott

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blakley Sr.

DR 17-030296

INTOXILYZER 8000
Location: YCSO PRESCOTT
Serial Number: 80-001019
Core Version: 8105.48
08/23/2017 21:40:25

Standard Lot#: 27016100A1
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 21:40:57
0.100 Cal Check 0.098 21:41:18
Air Blank 0.000 21:41:47

Preventative Maintenance
Performed -- Timer Reset