

DR18-035534

COPY

EXHIBIT G-2  
 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)  
 ARIZONA DEPARTMENT OF PUBLIC SAFETY  
 STANDARD QUALITY ASSURANCE PROCEDURES  
 INTOXILYZER MODEL 8000  
 STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO  
 DATE 9-26-18 TIME 22:58:00

INTOXILYZER SERIAL # 80-001019 LOCATION YCSO Prescott Jail

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
 Pour a standard alcohol concentration solution OR NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.097 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blakley Sr.

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 05218100A1  
 Exp 05-05-2020  
 Press 700 psi

DR 18-035534

INTOXILYZER 8000  
Location: YCSO PRESCOTT  
Serial Number: 80-001019  
Core Version: 8105.48  
09/26/2018 22:58:00

-----  
Standard Lot#: 05218100A1  
Last Changed By: S. BLAKLEY SR #L6861  
-----

QAS: S. BLAKLEY SR #L6861  
YAVAPAI CSO  
-----

Operator: S. BLAKLEY SR #L6861  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	22:58:32
0.100 Cal Check	0.097	22:58:53
Air Blank	0.000	22:59:22

Preventative Maintenance  
Performed -- Timer Reset