

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.
DATE 4-2-08 TIME 1815
INTOXILYZER SERIAL # 80-001095 LOCATION MAYER

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 098 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel J. Raiss #5-31/OAS #33026

DPS Form Exh G-2 (Rev 05-01)

NEW DRY GAS.
LOT 733201G
EXPIRES 12-3-09

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number: 80-001095
04/02/2008 18:16:30

Standard Lot#: 733201G
Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:16:57
Cal Check	0.098	18:17:16
Air Blank	0.000	18:17:46

April Q. Rain #5-31/QAS #33026
Operator's Signature

*NEW GAS

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co.S.O.
DATE 4-2-08 TIME 1730
INTOXILYZER SERIAL # 80001095 LOCATION MAYER

- OR 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- OR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
OR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- OR) 4. Air blank completed.
OR) 5. Calibration check completed. Test results 0. 098 AC.
OR) 6. Air blank completed.
OR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel J. Raiss #5-31 / QAS #33026

DPS Form Exh G-2 (Rev 05-01)

CALCK PRIOR TO DRY GAS CHANGE

INTOXILYZER 8000

Location: YCSO-MAYER

Serial Number: 80-001095

04/02/2008

18:12:42

Standard Lot#: 610201I

Last Changed By: D. RAISS #S-31

QAS: D. RAISS #3693

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:13:09
Cal Check	0.098	18:13:28
Air Blank	0.000	18:13:58

Daniel G. Reiss #5-31 / OAS #33026
Operator's Signature

*PRIOR

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI CO.S.O.
DATE 4-2-08 TIME 1730
INTOXILYZER SERIAL # 80-001095 LOCATION MAYER

OK 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- OK 1. Clock time check.
OK 2. Date check.

OPERATIONAL TESTS

- OK 1. Alcohol-free subject test result 0. 000 AC.
OK 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
OK 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
OK 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

OBTAINED AMBIENT FAIL x2 WHILE TRYING RFI. RESET INSTRUMENT AFTER
CONFIRMING NO AMBIENT SOURCES PRESENT. RFI WORKING PROPERLY ON TEST.
All OK.

SIGNATURE Daniel J. Raiss #3-31 / ORS #33026

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number : 80-001095
04/02/2008

18:03:22

Standard Lot#: 610201I
Last Changed By: D. RAISS #S-31

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP ,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:04:17
Diagnostic Test	Pass	18:04:45
Air Blank	0.000	18:05:12
0.100 Cal Check	0.099	18:05:31
Air Blank	0.000	18:06:00
Subject Test	0.080*	18:09:09
Air Blank	0.000	18:09:40
Air Blank	0.000	18:10:08
Subject Test	XXX**	18:10:25
Air Blank	0.000	18:10:59

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number : 80-001095
04/02/2008

17:59:34

Standard Lot#: 610201I
Last Changed By: D. RAISS #S-31

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP ,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	18:00:41
Air Blank	0.000	18:01:09

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number : 80-001095
04/02/2008

17:40:46

Standard Lot#: 610201I
Last Changed By: D. RAISS #S-31

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	AMB*	17:41:27
Air Blank	0.000	17:41:56

*Ambient Fail

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number : 80-001095
04/02/2008

17:38:40

Standard Lot#: 610201I
Last Changed By: D. RAISS #S-31

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	AMB*	17:39:23
Air Blank	0.000	17:39:51

*Ambient Fail

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number : 80-001095
04/02/2008

17:37:44

Standard Lot#: 610201I
Last Changed By: D. RAISS #S-31

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	17:38:07
Air Blank	0.000	17:38:35

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number : 80-001095
04/02/2008

17:32:11

Standard Lot#: 610201I
Last Changed By: D. RAISS #S-31

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:33:38
Diagnostic Test	Pass	17:34:06
Air Blank	0.000	17:34:33
0.100 Cal Check	0.099	17:34:52
Air Blank	0.000	17:35:21
Subject Test	0.000	17:36:02
Air Blank	0.000	17:36:29
Five Minute Wait	ABT*	
Air Blank	0.000	17:37:19

*Sequence Aborted

Not a Successfully Completed Test Sequence