

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAIS AGENCY YAVAPAI CO. S.O.
DATE 7-23-08 TIME 1540
INTOXILYZER SERIAL # 80-001095 LOCATION YCSO-MAYER

- DR) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- DR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- DR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- DR) 4. Air blank completed.
- DR) 5. Calibration check completed. Test results 0. 097^① AC, 097^② AC.
- DR) 6. Air blank completed.
- DR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Manil G Rains #5-31 / ORS #33026

DPS Form Exh G-2 (Rev 05-01)

DID 2 CALCS.

INTOXILYZER 8000

Location: YCSO-MAYER

Serial Number: 80-001095

07/23/2008

15:47:39

Standard Lot#: 733201G

Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	15:48:07
Cal Check	0.097	15:48:25
Air Blank	0.000	15:48:55

Daniel J. Raiss #5-31 / QAS #33026
Operator's Signature

INTOXILYZER 8000

Location: YCSO-MAYER

Serial Number: 80-001095

07/23/2008

15:43:07

Standard Lot#: 733201G

Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	15:43:35
Cal Check	0.097	15:43:53
Air Blank	0.000	15:44:23

Amil Q. Rain #5-31/QAS #33026
Operator's Signature