

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DANIEL McGINNIS AGENCY YCSO

DATE 11-14-08 TIME 2014

INTOXILYZER SERIAL # 80-001095 LOCATION MAYE SUB-STATION

- DM* 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- DM* 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- DM* 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- DM* 4. Air blank completed.
- DM* 5. Calibration check completed. Test results 0. 097 AC.
- DM* 6. Air blank completed.
- DM* 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Daniel McGinnis* 543/8086 OAS# 33426

INTOXILYZER 8000  
Location: YCSO-MAYER  
Serial Number: 80-001095  
11/14/2008 20:14:43

-----  
Standard Lot#: 733201G  
Last Changed By: D. RAISS #3693  
-----

QAS: D. MCGINNIS #8086  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	20:15:11
Cal Check	0.097	20:15:30
Air Blank	0.000	20:15:59


  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DANIEL M'GINNIS AGENCY YCSO  
DATE 11-14-08 TIME 2023  
INTOXILYZER SERIAL # 80-001095 LOCATION MAYER SUBSTATION

*DM* 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- DM* 1. Clock time check.  
*DM* 2. Date check.

OPERATIONAL TESTS

- DM* 1. Alcohol-free subject test result 0. 000 AC.  
*DM* 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
*DM* 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
*DM* 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

Amusement Fail on last cal. check  
ALL OK

SIGNATURE *Daniel McGinnis* 543/8086

INTOXILYZER 8000  
Location: YCSO-MAYER  
Serial Number : 80-001095  
11/14/2008

20:17:45

Standard Lot#: 733201G  
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086  
YAVAPAI CSO

Operator: D. MCGINNIS #8086  
YAVAPAI CSO

Subject: SQAP,TEST

DOB: 09/09/1963

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:18:46
Diagnostic Test	Pass	20:19:13
Air Blank	0.000	20:19:40
0.100 Cal Check	0.097	20:19:59
Air Blank	0.000	20:20:29
Subject Test	0.000	20:21:08
Air Blank	ABT*	20:21:18
Air Blank	RFI**	20:21:25

\*Sequence Aborted

\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO-MAYER  
Serial Number : 80-001095  
11/14/2008 20:23:20

Standard Lot#: 733201G  
Last Changed By: D. RAISS #3693

QAS: D. MCGINNIS #8086  
YAVAPAI CSO

Operator: D. MCGINNIS #8086  
YAVAPAI CSO

Subject: SQAP,TEST

DOB: 09/09/1963

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:24:21
Diagnostic Test	Pass	20:24:48
Air Blank	0.000	20:25:16
0.100 Cal Check	0.097	20:25:34
Air Blank	0.000	20:26:04
Subject Test	0.013*	20:29:12
Air Blank	0.000	20:29:45
Air Blank	0.000	20:30:13
Subject Test	XXX**	20:30:38
Air Blank	IPS***	20:30:47

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

\*\*\*Improper Sample


**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO-MAYER  
Serial Number: 80-001095  
11/14/2008 20:34:59

-----  
Standard Lot#: 733201G  
Last Changed By: D. RAISS #3693  
-----

QAS: D. MCGINNIS #8086  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	20:35:26
Cal Check	0.097	20:35:45
Air Blank	0.000	20:36:14

 543/8086 QAS #33426  
Operator's Signature

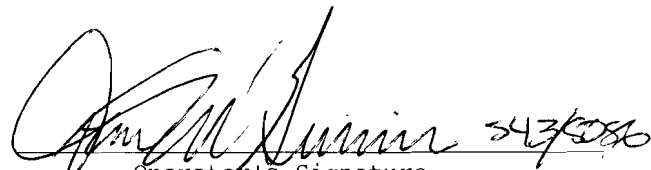
INTOXILYZER 8000  
Location: YCSO-MAYER  
Serial Number: 80-001095  
11/14/2008 20:33:02

-----  
Standard Lot#: 733201G  
Last Changed By: D. RAISS #3693  
-----

QAS: D. MCGINNIS #8086  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	AMB*	20:33:30
Air Blank	0.000	20:33:58

\*Ambient Fail

  
Operator's Signature