

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOORE AGENCY YAVAPAI CO. S.O.  
DATE 3/12/2013 TIME 0626  
INTOXILYZER SERIAL # 80-001095 LOCATION YCSO - MAYER

- (  ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (  ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (  ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (  ) 4. Air blank completed.
- (  ) 5. Calibration check completed. Test results 0. 095 AC.
- (  ) 6. Air blank completed.
- (  ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE LMoore 7486

INTOXILYZER 8000

Location: YCSO-MAYER

Serial Number: 80-001095

Core Version: 8105.48

03/12/2013

06:26:08

-----  
Standard Lot#: 32012100A1

Last Changed By: L. MOODY #7486  
-----

QAS: L. MOODY #7486

YAVAPAI CSO  
-----

Operator: L. MOODY #7486

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	06:26:41
0.100 Cal Check	0.095	06:27:01
Air Blank	0.000	06:27:31

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.  
DATE 3/11/2013 TIME 1411  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO DUI TRAILER

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution . 100 AC.  
OR  
~~Pour a standard alcohol concentration solution \_\_\_\_\_ AC. into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 095 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE LMoody 7486

INTOXILYZER 8000  
Instrument Initialization  
12:38:59 03/11/2013

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-001289  
Core Version: 8105.48  
03/11/2013 14:11:39

-----  
Standard Lot#: 13712100A2  
Last Changed By: L. MOODY #7486  
-----

QAS: L. MOODY #7486  
YAUAPA: CSO  
-----

Operator: L. MOODY #7486  
YAUAPA: CSO  
-----

Test	g/210L	Time
Air Blank	0.000	14:12:52
0.100 Cal Check	0.095	14:13:13
Air Blank	0.000	14:13:42

Preventative Maintenance  
Performed -- Timer Reset