

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KELLY WOHLWEND AGENCY YAVAPAI COUNTY S.O.

DATE 05/28/15 TIME 7308

INTOXILYZER SERIAL # 80-001095 LOCATION YCSO MAYHEW

() 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .10 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

() 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

() 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

() 4. Air blank completed.

() 5. Calibration check completed. Test results 0.096 AC.

() 6. Air blank completed.

() 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kelly Wohlwend 9871

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number: 80-001095
Core Version: 8105.48
05/28/2015 23:08:30

Standard Lot#: 33913100A3
Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811
YAVAPAI CO SO

Operator: K. WOHLWEND #9811
YAVAPAI CO SO

Test	g/210L	Time
Air Blank	0.000	23:09:30
0.100 Cal Check	0.096	23:09:51
Air Blank	0.000	23:10:20

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST KELLY WOHLWEND AGENCY YAVAPAI COUNTY S.O.
DATE 05/28/15 TIME 2312
INTOXILYZER SERIAL # 80-001095 LOCATION YCSO MAYER

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. ~~070~~ ¹⁰⁰ AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Kelly Wohlwend 9811

INTOXILYZER 8000

Location: YCSO-MAYER

Serial Number: 80-001095

Core Version: 8105.48

05/28/2015

23:12:42

Standard Lot#: 33913100A3

Last Changed By: L. MOODY

#7486

QAS: K. WOHLWEND

#9811

YAVAPAI CO SO

Operator: K. WOHLWEND

#9811

YAVAPAI CO SO

Subject: TEST,SOAP

DOB: 05/28/2000

Sex: M

Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:13:46
Diagnostic Test	Pass	23:14:21
Air Blank	0.000	23:14:48
0.100 Cal Check	0.097	23:15:09
Air Blank	0.000	23:15:39
Subject Test	0.000	23:16:01
Air Blank	0.000	23:16:28
Wait	ABT*	
Air Blank	IPS**	23:16:54

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number: 80-001095
Core Version: 8105.48
05/28/2015 23:17:11

Standard Lot#: 33913100A3
Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811
YAVAPAI CO SO

Operator: K. WOHLWEND #9811
YAVAPAI CO SO

Subject: TEST,SOAP
DOB: 05/28/2000
Sex: M Weight: 100

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:18:30
Diagnostic Test	Pass	23:19:05
Air Blank	0.000	23:19:32
0.100 Cal Check	0.096	23:19:53
Air Blank	0.000	23:20:23
Subject Test	0.029*	23:23:30
Air Blank	0.000	23:24:01
Air Blank	0.000	23:24:28
Subject Test	XXX**	23:24:50
Air Blank	RFI***	23:24:59

*Deficient Sample
**Invalid Sample
Begin new deprivation period
***RFI Detect

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST KELLY WOHLWEND AGENCY YAVAPAI COUNTY S.O.
DATE 05/28/15 TIME 2325
INTOXILYZER SERIAL # 80-001095 LOCATION YCSO MAYER

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .10 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 0% AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Kelly Wohlwend

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number: 80-001095
Core Version: 8105.48
05/28/2015 23:25:21

Standard Lot#: 33913100A3
Last Changed By: L. MOODY #7486

QAS: K. WOHLWEND #9811
YAVAPAI CO SO

Operator: K. WOHLWEND #9811
YAVAPAI CO SO

Test	g/210L	Time
Air Blank	0.000	23:25:51
0.100 Cal Check	0.096	23:26:11
Air Blank	0.000	23:26:41

Preventative Maintenance
Performed -- Timer Reset