

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office

DATE 11/18/2017 TIME 1446

INTOXILYZER SERIAL # 80-001095 LOCATION YCSO - Mayer

- (1) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (2) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (3) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (4) 4. Air blank completed.
- (5) 5. Calibration check completed. Test results 0. 099 AC.
- (6) 6. Air blank completed.
- (7) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

17-4489

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number: 80-001095
Core Version: 8105.48
11/18/2017 14:46:06

Standard Lot#: 06717100A5
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	14:46:35
0.100 Cal Check	0.099	14:46:55
Air Blank	0.000	14:47:25

Preventative Maintenance
Performed -- Timer Reset