

17-043599

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office

DATE 12/06/2017 TIME 1403

INTOXILYZER SERIAL # 80-001095 LOCATION YCSO - Mayer

(Handwritten initials)

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

(Handwritten initials)

2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(Handwritten initials)

4. Air blank completed.

5. Calibration check completed. Test results 0.099 AC.

(Handwritten initials)

6. Air blank completed.

7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *(Signature)* 7486

17-043599



ARIZONA DEPARTMENT OF PUBLIC SAFETY
INTOXILYZER MAINTENANCE REPORT

CR NO. I800010950005

Page 1 of 1

AGENCY Yavapai County Sheriff's Office
Prescott, AZ 86301
OFFICER MOODY, #7486
DATE November 30, 2017



INSTRUMENT

80001095

INSTRUMENT LOCATION

Yavapai County Sheriff's Office

MAINTENANCE REQUESTED

Preventative Maintenance

MAINTENANCE PERFORMED

Preventative Maintenance Performed:

Inspection, Calibration, Verification

ROBERT W. STEPHENSON, #5742, Forensic Scientist
Central Regional Crime Laboratory
2102 W. Encanto Blvd., Phoenix, AZ 85009
602-223-2114