

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office

DATE 04/20/2018 TIME 1353

INTOXILYZER SERIAL # 80-001095 LOCATION YCSO - Mayer

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. ~~Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 098 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody #7486

18-014175

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number: 80-001095
Core Version: 8105.48
04/20/2018 13:53:19

Standard Lot#: 06717100A5
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 13:53:48
0.100 Cal Check 0.098 13:54:09
Air Blank 0.000 13:54:38

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office

DATE 04/20/2018 TIME 1355

INTOXILYZER SERIAL # 80-001095 LOCATION YCSO - Mayer

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.

2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.

2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed

3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed

4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

900 psi

SIGNATURE Logan Moody 7486

INTOXILYZER 8000
 Location: YCSO-MAYER
 Serial Number: 80-001095
 Core Version: 8105.48
 04/20/2018

13:55:59

Standard Lot#: 06717100A5
 Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
 YAVAPAI CSO

Operator: L. MOODY #7486
 YAVAPAI CSO

Subject: SQAP,TEST
 DOB: 01/01/1990
 Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:56:46
Diagnostic Test	Pass	13:57:21
Air Blank	0.000	13:57:49
0.100 Cal Check	0.098	13:58:09
Air Blank	0.000	13:58:39
Subject Test	0.000	13:59:06
Air Blank	ABT*	13:59:14
Air Blank	RFI**	13:59:22

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
 Location: YCSO-MAYER
 Serial Number: 80-001095
 Core Version: 8105.48
 04/20/2018

14:00:25

Standard Lot#: 06717100A5
 Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
 YAVAPAI CSO

Operator: L. MOODY #7486
 YAVAPAI CSO

Subject: SQAP, TEST
 DOB: 01/01/1990
 Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:01:12
Diagnostic Test	Pass	14:01:47
Air Blank	0.000	14:02:14
0.100 Cal Check	0.098	14:02:35
Air Blank	0.000	14:03:04
Subject Test	0.154*	14:06:12
Air Blank	0.000	14:06:43
Air Blank	0.000	14:07:11
Subject Test	XXX**	14:08:56
Air Blank	IPS***	14:09:04

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***Improper Sample

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office
DATE 04/20/2018 TIME 1411
INTOXILYZER SERIAL # 80-001095 LOCATION YCSO - Mayer

- (u) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (u) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (u) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (u) 4. Air blank completed.
- (u) 5. Calibration check completed. Test results 0. 099 AC.
- (u) 6. Air blank completed.
- (u) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody #7486

INTOXILYZER 8000
Location: YCSO-MAYER
Serial Number: 80-001095
Core Version: 8105.48
04/20/2018 14:11:45

Standard Lot#: 06717100A5
Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	14:12:13
0.100 Cal Check	0.099	14:12:34
Air Blank	0.000	14:13:04

Preventative Maintenance
Performed -- Timer Reset