

18-036527

**EXHIBIT G-2**  
**THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)**  
**ARIZONA DEPARTMENT OF PUBLIC SAFETY**  
**STANDARD QUALITY ASSURANCE PROCEDURES**  
**INTOXILYZER MODEL 8000**  
**STANDARD CALIBRATION CHECK PROCEDURE**

QA SPECIALIST Logan Moody #7486 AGENCY Yavapai County Sheriff's Office  
 DATE 10/04/2018 TIME 1548  
 INTOXILYZER SERIAL # 80-001095 LOCATION YCSO - Mayer

- (10) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
 OR  
~~Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (6) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (6) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (6) 4. Air blank completed.
- (6) 5. Calibration check completed. Test results 0. 098 AC.
- (6) 6. Air blank completed.
- (6) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody #7486

18-036527

INTOXILYZER 8000  
Location: YCSO-MAYER  
Serial Number: 80-001095  
Core Version: 8105.48  
10/04/2018 15:48:05

-----  
Standard Lot#: 06717100A5  
Last Changed By: L. MOODY #7486  
-----

QAS: L. MOODY #7486  
YAVAPAI CSO  
-----

Operator: L. MOODY #7486  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	15:48:40
0.100 Cal Check	0.098	15:49:01
Air Blank	0.000	15:49:31

Preventative Maintenance  
Performed -- Timer Reset