

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Sgt. D. Jacobs AGENCY Camp Verde Marshals
DATE 3/6/17 TIME 1111
INTOXILYZER SERIAL # 80-001096 LOCATION CVMO Evidence room

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.105 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0.100 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  CV08

INTOXILYZER 8000
Location: CAMP VERDE MO
Serial Number: 80-001096
Core Version: 8105.48
03/06/2017 11:11:21

Standard Lot#: 32316100A1
Last Changed By: D. JACOBS #CV08

QAS: D. JACOBS #CV08
CAMP VERDE MO

Operator: D. JACOBS #CV08
CAMP VERDE MO

Test	g/210L	Time
Air Blank	0.000	11:12:20
0.100 Cal Check	0.100	11:12:41
Air Blank	0.000	11:13:11

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sgt. D. Jacobs AGENCY Camp Verde Marshals
DATE 3/6/17 TIME 1113
INTOXILYZER SERIAL # 80-001096 LOCATION CVMO Evidence Room

(f) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (f) 1. Clock time check.
(x) 2. Date check.

OPERATIONAL TESTS

- (x) 1. Alcohol-free subject test result 0. 000 AC.
(x) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(x) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(x) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES X NO

COMMENTS

SIGNATURE  CV08

INTOXILYZER 8000
Location: CAMP VERDE MO
Serial Number: 80-001096
Core Version: 8105.48
03/06/2017

11:17:08

Standard Lot#: 32316100A1
Last Changed By: D. JACOBS #CV08

QAS: D. JACOBS #CV08
CAMP VERDE MO

Operator: D. JACOBS #CV08
CAMP VERDE MO

Subject: TEST,TEST
DOB: 03/16/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:18:12
Diagnostic Test	Pass	11:18:46
Air Blank	0.000	11:19:13
0.100 Cal Check	0.100	11:19:34
Air Blank	0.000	11:20:04
Subject Test	0.072*	11:23:11
Air Blank	0.000	11:23:43
Air Blank	0.000	11:24:11
Subject Test	XXX**	11:25:08
Air Blank	RFI***	11:25:21

*Deficient Sample
**Invalid Sample
Begin new deprivation period
***RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: CAMP VERDE MO
Serial Number: 80-001096
Core Version: 8105.48
03/06/2017

11:13:25

Standard Lot#: 32316100A1
Last Changed By: D. JACOBS #CV08

QAS: D. JACOBS #CV08
CAMP VERDE MO

Operator: D. JACOBS #CV08
CAMP VERDE MO

Subject: TEST,TEST
DOB: 03/16/1990
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:14:36
Diagnostic Test	Pass	11:15:10
Air Blank	0.000	11:15:38
0.100 Cal Check	0.100	11:15:59
Air Blank	0.000	11:16:28
Subject Test	0.000	11:16:47
Air Blank	ABT*	11:16:54
Air Blank	IPS**	11:17:02

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Sgt. D. Jacobs AGENCY Camp Verde Marshals
DATE 3/6/17 TIME 1128
INTOXILYZER SERIAL # 80-001096 LOCATION CVMO Evidence Room

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 099 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  CV08

INTOXILYZER 8000
Location: CAMP VERDE MO
Serial Number: 80-001096
Core Version: 8105.48
03/06/2017 11:28:05

Standard Lot#: 32316100A1
Last Changed By: D. JACOBS #CV08

QAS: D. JACOBS #CV08
CAMP VERDE MO

Operator: D. JACOBS #CV08
CAMP VERDE MO

Test	g/210L	Time
Air Blank	0.000	11:28:40
0.100 Cal Check	0.099	11:29:01
Air Blank	0.000	11:29:31

Preventative Maintenance
Performed -- Timer Reset