

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Jacobs CV08 AGENCY Camp Verde Marshals office  
DATE 4/18/17 TIME 1349  
INTOXILYZER SERIAL # 80-001096 LOCATION CVMO DUI Trailer

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (+) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(+) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(+) 4. Air blank completed.  
(+) 5. Calibration check completed. Test results 0. 098 AC.  
(+) 6. Air blank completed.  
(+) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] Sgt. CV08

