

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL, 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Deborah Hughes AGENCY Camp Verde Marshal's Ofc
DATE 9-27-17 TIME 0840
INTOXILYZER SERIAL # 80-001096 LOCATION CUMV Booking

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START" Contact QAS 31 Day Prev Maint
(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(X) 4. Air blank completed.
(X) 5. Calibration check completed. Test results 0.100 AC.
(X) 6. Air blank completed.
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] #63

