

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Sgt. D. Jacobs AGENCY Camp Verde Marshals office  
DATE 3/2/18 TIME \_\_\_\_\_  
INTOXILYZER SERIAL # 80-001096 LOCATION CVMO Booking

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 100 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE \_\_\_\_\_

CV08 D. Jacobs

INTOXILYZER 8000

Location: CAMP VERDE MO

Serial Number: 80-001096

Core Version: 8105.48

03/02/2018

19:51:46

-----  
Standard Lot#: 32316100A1

Last Changed By: D. JACOBS #CV08  
-----

QAS: D. JACOBS #CV08

CAMP VERDE MO  
-----

Operator: D. JACOBS #CV08

CAMP VERDE MO  
-----

Test	g/210L	Time
Air Blank	0.000	19:52:18
0.100 Cal Check	0.100	19:52:39
Air Blank	0.000	19:53:09

Preventative Maintenance  
Performed -- Timer Reset