

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co.S.O.
DATE 4-2-08 TIME 1932
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO-VERDE JAIL

- (OR) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution X AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (OR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (OR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (OR) 4. Air blank completed.
- (OR) 5. Calibration check completed. Test results 0. 100 AC.
- (OR) 6. Air blank completed.
- (OR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel J Raiss #5-31/QRS #33026

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

04/02/2008

19:50:28

Standard Lot#: 633901I

Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	19:50:55
Cal Check	0.100	19:51:14
Air Blank	0.000	19:51:44

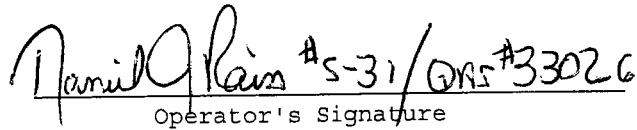
 #S-31/ORS #33026
Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.
DATE 4-2-08 TIME 1932
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO-VERDE JAIL

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 100 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

ALL OK

SIGNATURE

Daniel Q Raiss #5-31 / ORS #33026

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number : 80-001222
04/02/2008 19:42:30

Standard Lot#: 633901I
Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:43:22
Diagnostic Test	Pass	19:43:49
Air Blank	0.000	19:44:17
0.100 Cal Check	0.100	19:44:36
Air Blank	0.000	19:45:06
Subject Test	0.172*	19:48:14
Air Blank	0.000	19:48:46
Air Blank	0.000	19:49:13
Subject Test	XXX**	19:49:33
Air Blank	0.000	19:50:06

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number : 80-001222
04/02/2008 19:41:12

Standard Lot#: 633901I
Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Subject: SQAP, TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	19:41:45
Air Blank	0.000	19:42:13

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number : 80-001222
04/02/2008

19:39:31

Standard Lot#: 633901I
Last Changed By: D. RAISS #3693

QAS: D. RAISS #3693
YAVAPAI CSO

Operator: D. RAISS #S-31
YAVAPAI CSO

Subject: SQAP,TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	19:40:32
Air Blank	0.000	19:41:00

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number : 80-001222

04/02/2008

19:33:42

Standard Lot#: 633901I

Last Changed By: D. RAISS

#3693

QAS: D. RAISS

#3693

YAVAPAI CSO

Operator: D. RAISS

#3693

YAVAPAI CSO

Subject: SQAP,TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:35:09
Diagnostic Test	Pass	19:35:36
Air Blank	0.000	19:36:04
0.100 Cal Check	0.100	19:36:23
Air Blank	0.000	19:36:52
Subject Test	0.000	19:37:53
Air Blank	0.000	19:38:21
Five Minute Wait	ABT*	
Air Blank	0.000	19:39:16

*Sequence Aborted

Not a Successfully Completed Test Sequence