

**EXHIBIT G-2**  
**THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)**  
**ARIZONA DEPARTMENT OF PUBLIC SAFETY**  
**STANDARD QUALITY ASSURANCE PROCEDURES**  
**INTOXILYZER MODEL 8000**  
**STANDARD CALIBRATION CHECK PROCEDURE**

QA SPECIALIST DAN RAISS AGENCY YAVAPAI CO.S.O.  
 DATE 10-14-08 TIME 1730  
 INTOXILYZER SERIAL # 80-001222 LOCATION YCSO-VERDE JAIL

- (OR) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
 OR  
 Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (OR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (OR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (OR) 4. Air blank completed.
- (OR) 5. Calibration check completed. Test results 0. 1.02 AC.
- (OR) 6. Air blank completed.
- (OR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel Q Rains #5-31/QAS #33026

INTOXILYZER 8000  
Location: YCSO-VERDE JAIL  
Serial Number: 80-001222  
10/14/2008 17:45:20

-----  
Standard Lot#: 804301I  
Last Changed By: D. RAISS #3693  
-----

QAS: D. RAISS #3693  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	17:45:47
Cal Check	0.102	17:46:06
Air Blank	0.000	17:46:36

*Amil Q. Kam* #5-31/QAS #33026  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 10-14-08 TIME 1730  
INTOXILYZER SERIAL # 80-001095 LOCATION YCSO-VERDE JAIL

(OK) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (OK) 1. Clock time check.  
(OK) 2. Date check.

OPERATIONAL TESTS

- (OK) 1. Alcohol-free subject test result 0. 000 AC.  
(OK) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(OK) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(OK) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 102 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS All OK.

SIGNATURE Daniel Q. Raim #5-31/DAS #33026

INTOXILYZER 8000  
Location: YCSO-VERDE JAIL  
Serial Number : 80-001222  
10/14/2008 17:37:31

---

Standard Lot#: 804301I  
Last Changed By: D. RAISS #3693

---

QAS: D. RAISS #3693  
YAVAPAI CSO

---

Operator: D. RAISS #3693  
YAVAPAI CSO

---

Subject: SQAP,TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	17:38:18
Diagnostic Test	Pass	17:38:45
Air Blank	0.000	17:39:13
0.100 Cal Check	0.102	17:39:31
Air Blank	0.000	17:40:01
Subject Test	0.073*	17:43:10
Air Blank	0.000	17:43:42
Air Blank	0.000	17:44:09
Subject Test	XXX**	17:44:31
Air Blank	0.000	17:45:05

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: YCSO-VERDE JAIL  
Serial Number : 80-001222  
10/14/2008 17:36:30

---

Standard Lot#: 804301I  
Last Changed By: D. RAISS #3693

---

QAS: D. RAISS #3693  
YAVAPAI CSO

---

Operator: D. RAISS #3693  
YAVAPAI CSO

---

Subject: SQAP ,TEST  
DOB: 01/01/1901  
Sex: M Weight: 1

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	17:36:56
Air Blank	0.000	17:37:25

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO-VERDE JAIL  
Serial Number : 80-001222  
10/14/2008 17:35:37

---

Standard Lot#: 804301I  
Last Changed By: D. RAISS #3693

---

QAS: D. RAISS #3693  
YAVAPAI CSO

---

Operator: D. RAISS #3693  
YAVAPAI CSO

---

Subject: SQAP,TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	17:35:58
Air Blank	0.000	17:36:26

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: YCSO-VERDE JAIL  
Serial Number : 80-001222  
10/14/2008 17:30:14

---

Standard Lot#: 804301I  
Last Changed By: D. RAISS #3693

---

QAS: D. RAISS #3693  
YAVAPAI CSO

---

Operator: D. RAISS #3693  
YAVAPAI CSO

---

Subject: SQAP,TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	17:31:41
Diagnostic Test	Pass	17:32:08
Air Blank	0.000	17:32:36
0.100 Cal Check	0.101	17:32:55
Air Blank	0.000	17:33:24
Subject Test	0.000	17:33:40
Air Blank	0.000	17:34:07
Five Minute Wait	ABT*	
Air Blank	0.000	17:35:32

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

---