

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YUWAPAI Co. S.O.
DATE 2-24-10 TIME 1330
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO-VERDE JAIL

- (OR) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (OR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (OR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (OR) 4. Air blank completed.
- (OR) 5. Calibration check completed. Test results 0. 099 AC.
- (OR) 6. Air blank completed.
- (OR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel J. Raiss #N-11/OAS #33026

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
02/24/2010 13:49:25

Standard Lot#: 6852215
Last Changed By: D. MCGINNIS #8086

OAS: D. RAISS #3693
YUWAPAI CSO

Operator: D. RAISS #3693
YUWAPAI CSO

Test	g/210L	Time
Air Blank	0.000	13:49:55
0.100 Cal. Check	0.099	13:50:16
Air Blank	0.000	13:50:45

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.
DATE 2-24-10 TIME 1330
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO-VERDE JAIL

OR 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- OR 1. Clock time check.
OR 2. Date check.

OPERATIONAL TESTS

- OR 1. Alcohol-free subject test result 0. 000 AC.
OR 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
OR 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
OR 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SOFTWARE UPGRADE. SAME DRY GAS. ALL OK.

SIGNATURE

Daniel P. Rain #N-11 / OAS #33026

INTOXILYZER 8000
Instrument Initialization
13:26:01 02/24/2010

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
02/24/2010 13:33:54

Standard Lot#: 6852215
Last Changed By: D. MCGINNIS #8086

OAS: D. RAISS #3693
YAUAPAI CSO

Operator: D. RAISS #3693
YAUAPAI CSO

Subject: SOAP, TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:34:47
Diagnostic Test	Pass	13:35:25
Air Blank	0.000	13:35:52
0.100 Cal Check	0.100	13:36:13
Air Blank	0.000	13:36:43
Subject Test	0.000	13:37:02
Air Blank	0.000	13:37:30
Wait	66T*	
Air Blank	0.000	13:38:13

*Sequence Aborted

Not a Successfully
Completed 1
Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
02/24/2010 13:38:22

Standard Lot#: 6852215
Last Changed By: D. MCGINNIS #8086

OAS: D. RAISS #3693
YAUAPAI CSO

Operator: D. RAISS #3693
YAUAPAI CSO

Subject: SOAP, TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	13:38:46
Air Blank	0.000	13:39:15

*Improper Sample

Not a Successfully
Completed 2
Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
02/24/2010 13:39:23

Standard Lot#: 6852215
Last Changed By: D. MCGINNIS #8086

OAS: D. RAISS #3693
YAUAPAI CSO

Operator: D. RAISS #3693
YAUAPAI CSO

Subject: SOAP, TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:39:46
Air Blank	0.000	13:40:14

*RFI Detect

Not a Successfully
Completed 3
Test Sequence

13:40:22

Standard Lot#: 6852215
Last Changed By: D. MCGINNIS #8086

OAS: D. RAISS #3693
YAUAPAI CSO

Operator: D. RAISS #3693
YAUAPAI CSO

Subject: SOAP, TEST,
DOB: 01/01/1901
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:41:05
Diagnostic Test	Pass	13:41:42
Air Blank	0.000	13:42:10
0.100 Cal Check	0.100	13:42:30
Air Blank	0.000	13:43:00
Subject Test	0.189*	13:46:08
Air Blank	0.000	13:46:40
Air Blank	0.000	13:47:08
Subject Test	XXX**	13:47:32
Air Blank	0.000	13:48:06

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully
Completed 4
Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI CO. S.O.
DATE 2-24-10 TIME 1050
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO-VERDE JAIL

- DR 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- DR 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- DR 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- DR 4. Air blank completed.
- DR 5. Calibration check completed. Test results 0. 100 AC.
- DR 6. Air blank completed.
- DR 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel Q. Raiss #N-11 / QAS #33026

DPS Form Exh G-2 (Rev 05-01)

SOFTWARE UPGRADE
CAL. CHECK

INTOXILYZER 8000
Instrument Initialization
10:27:35 02/24/2010

INTOXILYZER 8000
Instrument Initialization
10:29:03 02/24/2010

INTOXILYZER 8000
Instrument Initialization
10:29:03 02/24/2010
TEST CHARGED BY: D. RAISS #33026
GAS: D. RAISS #33026
YAVAPAI CSO

Operator: D. RAISS #3693
YAVAPAI CSO

Test	gr/210L	Time
Air Blank	0.000	10:50:38
0.100 Cal Check	0.100	10:50:59
Air Blank	0.000	10:51:28

Preventative Maintenance
Performed -- Timer Reset