

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (3)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Daniel M'Crillis AGENCY YCSO
DATE 1-12-11 TIME 0503
INTOXILYZER SERIAL # 80-001222 LOCATION Verde Trails

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 10 AC
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. ~~Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 100 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Daniel M'Crillis 543/QAS# 33426

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
01/12/2011 05:03:16

Standard Lot#: 712322
Last Changed By: D. MCGINNIS #8086

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	05:03:46
0.100 Cal Check	0.100	05:04:07
Air Blank	0.000	05:04:37

Preventative Maintenance
Performed -- Timer Reset