

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (1)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. M. Minnis AGENCY YCSO
DATE 7-6-11 TIME 0445
INTOXILYZER SERIAL # 80-001222 LOCATION Verde Jail

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 10 AC
OR
Pour a ~~standard alcohol concentration solution~~ AC into a clean dry simulator and assemble the simulator. Ensure that a ~~tight seal has been made~~. Turn on the simulator and allow temperature to reach ~~34°C ± 0.2°C~~
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.097 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

D. M. Minnis 343/QAS # 23426

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
07/06/2011 04:45:15

Standard Lot#: 35710100A2
Last Changed By: D. MCGINNIS #8086

QAS: D. MCGINNIS #8086
YAVAPAI CSO

Operator: D. MCGINNIS #8086
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	04:46:31
0.100 Cal Check	0.097	04:46:52
Air Blank	0.000	04:47:21

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. McGinnis AGENCY YCSO
DATE 7-6-11 TIME 0447
INTOXILYZER SERIAL # 80-001227 LOCATION Verde Jail

DM 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

DM 1. Clock time check.
DM 2. Date check.

OPERATIONAL TESTS

DM 1. Alcohol-free subject test result 0. 000 AC.
DM 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
DM 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
DM 4. Standard Calibration Check standard 0 10 AC. Result 0. 097 AC

Instrument is operating properly and accurately. YES DM NO _____

COMMENTS

All OK

SIGNATURE

D. McGinnis 543/QAS# 3342C

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

07/06/2011

04:47:40

Standard Lot#: 35710100A2

Last Changed By: D. MCGINNIS

#8086

QAS: D. MCGINNIS

#8086

YAVAPAI CSO

Operator: D. MCGINNIS

#8086

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 09/09/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	04:48:45
Diagnostic Test	Pass	04:49:19
Air Blank	0.000	04:49:46
0.100 Cal Check	0.097	04:50:07
Air Blank	0.000	04:50:36
Subject Test	0.000	04:50:57
Air Blank	ABT*	04:51:06
Air Blank	RFI**	04:51:14

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

07/06/2011

04:51:27

Standard Lot#: 35710100A2

Last Changed By: D. MCGINNIS #8086

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Operator: D. MCGINNIS #8086

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 09/09/1990

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	04:52:22
Diagnostic Test	Pass	04:52:56
Air Blank	0.000	04:53:24
0.100 Cal Check	0.098	04:53:44
Air Blank	0.000	04:54:14
Subject Test	0.207*	04:57:22
Air Blank	0.000	04:57:57
Air Blank	0.000	04:58:25
Subject Test	XXX**	04:58:50
Air Blank	IPS***	04:58:58

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

07/06/2011

04:59:32

Standard Lot#: 35710100A2

Last Changed By: D. MCGINNIS #8086

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Operator: D. MCGINNIS #8086

YAVAPAI CSO

Test g/210L Time

Air Blank AMB* 05:00:02
Air Blank 0.000 05:00:30

*Ambient Fail

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

07/06/2011

05:00:38

Standard Lot#: 35710100A2

Last Changed By: D. MCGINNIS #8086

QAS: D. MCGINNIS #8086

YAVAPAI CSO

Operator: D. MCGINNIS #8086

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	05:01:08
0.100 Cal Check	0.097	05:01:29
Air Blank	0.000	05:01:58

Preventative Maintenance
Performed -- Timer Reset