

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST JEFFREY R. STAEGER AGENCY YAVAPAI COUNTY SHERIFF'S OFFICE
DATE 10-19-2011 TIME 2230

INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE DETENTION CENTER

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.094 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] # 3566

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

10/19/2011

22:29:44

Standard Lot#: 35710100A2

Last Changed By: J. JAEGER #3566

QAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	22:30:16
0.100 Cal Check	0.099	22:30:37
Air Blank	0.000	22:31:07

Preventative Maintenance
Performed -- Timer Reset

JJA #3566

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST JEFFREY R. JAEGER AGENCY YAVAPAI COUNTY SHERIFF'S OFFICE
DATE 10-19-2011 TIME 2240

INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE DETOUR CENTER

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0.000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0.100 AC. Result 0.098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

COMPLETED ANOTHER ALCOHOL-FREE TEST AFTER SECOND INVALID SAMPLE TEST

SECOND INVALID SAMPLE = 2255

SECOND ALCOHOL-FREE = 2259

INSTRUMENT IS OPERATING PROPERLY AND ACCURATELY

SIGNATURE

 #3566

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
10/19/2011

22:40:10

Standard Lot#: 35710100A2
Last Changed By: J. JAEGER

#3566

OAS: J. JAEGER
YAVAPAI CSO

#3566

Operator: J. JAEGER
YAVAPAI CSO

#3566

Subject: SQAP, TEST
DOB: 06/28/1985
Sex: M

Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:42:07
Diagnostic Test	Pass	22:42:41
Air Blank	0.000	22:43:08
0.100 Cal Check	0.099	22:43:29
Air Blank	0.000	22:43:59
Subject Test	0.000	22:44:21
Air Blank	0.000	22:44:48
Wait	ABT*	
Air Blank	RFI**	22:45:42

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
10/19/2011

22:46:40

Standard Lot#: 35710100A2
Last Changed By: J. JAEGER #3566

OAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 06/28/1985
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:47:49
Diagnostic Test	Pass	22:48:23
Air Blank	0.000	22:48:50
0.100 Cal Check	0.100	22:49:11
Air Blank	0.000	22:49:41
Subject Test	XXXX*	22:50:08
Air Blank	IPS**	22:50:33

*Invalid Sample
Begin new deprivation period
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
10/19/2011

22:50:50

Standard Lot#: 35710100A2 #3566
Last Changed By: J. JAEGER

OAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Subject: SQAP, TEST
DOB: 06/28/1985
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	AMB*	22:51:51
Air Blank	0.000	22:52:19

*Ambient Fail

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
10/19/2011

22:52:31

Standard Lot#: 35710100A2
Last Changed By: J. JAEGER #3566

QAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 06/28/1985
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:53:31
Diagnostic Test	Pass	22:54:05
Air Blank	0.000	22:54:32
0.100 Cal Check	0.099	22:54:53
Air Blank	0.000	22:55:23
Subject Test	XXX*	22:55:41
Air Blank	0.000	22:56:10

*Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
10/19/2011

22:56:17

Standard Lot#: 35710100A2
Last Changed By: J. JAEGER #3566

QAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 06/28/1985
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:57:40
Diagnostic Test	Pass	22:58:14
Air Blank	0.000	22:58:42
0.100 Cal Check	0.099	22:59:02
Air Blank	0.000	22:59:32
Subject Test	0.000	22:59:52
Air Blank	0.000	23:00:19
Wait		
Air Blank	0.000	23:05:20
Subject Test	0.000	23:05:35
Air Blank	ABT*	23:05:52
Air Blank	0.000	23:06:20

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
10/19/2011

23:06:25

Standard Lot#: 35710100A2
Last Changed By: J. JAEGER #3566

QAS: J. JAEGER #3566
YAVAPAI CSO

Operator: J. JAEGER #3566
YAVAPAI CSO

Subject: SQAP,TEST

DOB: 06/28/1985

Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:07:29
Diagnostic Test	Pass	23:08:03
Air Blank	0.000	23:08:31
0.100 Cal Check	0.099	23:08:51
Air Blank	0.000	23:09:21
Subject Test	0.020*	23:12:29
Air Blank	0.000	23:12:59
Air Blank	0.000	23:13:27
Subject Test	ABT**	23:14:12
Air Blank	0.000	23:14:40

*Deficient Sample

**Sequence Aborted

Not a Successfully Completed Test Sequence


EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SEFFREY R. STAEGER AGENCY YAVAPAI COUNTY SHERIFF'S OFFICE
DATE 10-19-2011 TIME 2320

INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE DETENTION CENTER

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.098 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  #3566

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

10/19/2011

23:20:57

Standard Lot#: 35710100A2

Last Changed By: J. JAEGER #3566

OAS: J. JAEGER #3566

YAVAPAI CSO

Operator: J. JAEGER #3566

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:21:27
0.100 Cal Check	0.098	23:21:48
Air Blank	0.000	23:22:18

Preventative Maintenance
Performed -- Timer Reset

J. Jaeger #3566