

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOORE AGENCY YAVAPAI CO. S.O.
DATE 11-20-2011 TIME 2242
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0.098 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE L. MOORE 27486

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

11/20/2011

22:42:43

Standard Lot#: 15811100A1

Last Changed By: L. MOODY #7486

QAS: L. MOODY #7486

YAVAPAI CSO

Operator: L. MOODY #7486

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 22:43:15
0.100 Cal Check 0.098 22:43:35
Air Blank 0.000 22:44:05

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.
DATE 11-20-2011 TIME 2233
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. _____ AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

DPS Form Exh G-2 (Rev 05-01)

RFI DETECT

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

11/20/2011

22:33:54

Standard Lot#: 15811100A1

Last Changed By: L. MOODY #7486

QAS: J. JAEGER #3566

YAVAPAI CSO

Operator: L. MOODY #7486

YAVAPAI CSO

Test	g/210L	Time
Air Blank	RFI*	22:34:28
Air Blank	0.000	22:34:57

*RFI Detect

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

11/20/2011

22:39:57

Standard Lot#: 15811100A1

Last Changed By: L. MOODY #7486

QAS: J. JAEGER #3566

YAVAPAI CSO

Operator: L. MOODY #7486

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	22:40:34
0.100 Cal Check	RFI*	22:40:46
Air Blank	0.000	22:41:17

*RFI Detect

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.

DATE 11-20-2011 TIME 2220

INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. _____ AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE L. MOODY 11/20/11

DPS Form Exh G-2 (Rev 05-01)

STANDARD CHANGE .100

LOT: 15811100A1

EXP: 8/1/2013

DSP FAIL X2 ON CAL CHECK

RECYCLED POWER AND STARTED NEW CAL CHECK