

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST CARLOS RODINA AGENCY YAVAPAI CO. SO.
DATE 3-14-12 TIME ~~1752~~ 2001
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.00 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.091 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Carlos Rodina* # 8528

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

03/14/2012

20:01:29

Standard Lot#: 15811100A1

Last Changed By: C. GODINA #8528

QAS: C. GODINA #8528

YAVAPAI CSO

Operator: C. GODINA #8528

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	20:02:28
0.100 Cal Check	0.099	20:02:49
Air Blank	0.000	20:03:19

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST CARLOS GODINA AGENCY YAVAPAI CO. SO
DATE 3-14-12 TIME 2021
INTOXILYZER SERIAL # 80-001222 LOCATION Camp Verde Jail

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed. 0.100
() 4. Standard Calibration Check standard 0. 0.100 / AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Deputy Carlos Godina

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

03/14/2012

20:21:07

Standard Lot#: 15811100A1

Last Changed By: C. GODINA #8528

QAS: C. GODINA #8528

YAVAPAI CSO

Operator: C. GODINA #8528

YAVAPAI CSO

Subject: DOE, JOHN

DOB: 07/07/1970

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:22:03
Diagnostic Test	Pass	20:22:37
Air Blank	0.000	20:23:04
0.100 Cal Check	0.099	20:23:25
Air Blank	0.000	20:23:55
Subject Test	0.000	20:24:31
Air Blank	0.000	20:24:58
Wait	ABT*	
Air Blank	IPS**	20:25:27

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
03/14/2012

20:26:09

Standard Lot#: 15811100A1
Last Changed By: C. GODINA #8528

QAS: C. GODINA #8528
YAVAPAI CSO

Operator: C. GODINA #8528
YAVAPAI CSO

Subject: DOE,JOHN
DOB: 07/07/1970
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:27:04
Diagnostic Test	Pass	20:27:38
Air Blank	0.000	20:28:05
0.100 Cal Check	0.098	20:28:26
Air Blank	0.000	20:28:56
Subject Test	0.000*	20:32:03
Air Blank	0.000	20:32:31
Air Blank	0.000	20:32:58
Subject Test	RGE**	20:33:29
Air Blank	RFI***	20:33:41

*Deficient Sample

**Range Exceeded

***RFI Detect

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST CARLOS RODINA AGENCY YAVAPAI CO. SO
DATE 3-14-12 TIME ~~2038~~ 2038
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Carlos Rodina

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

03/14/2012

20:38:43

Standard Lot#: 15811100A1

Last Changed By: C. GODINA #8528

QAS: C. GODINA #8528

YAVAPAI CSO

Operator: C. GODINA #8528

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 20:39:13
0.100 Cal Check 0.098 20:39:34
Air Blank 0.000 20:40:04

Preventative Maintenance
Performed -- Timer Reset