

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RT POOLE AGENCY YAVAPAI COUNTY S.O
DATE 2/10/13 TIME 1523
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START" STAND BY MODE
(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(X) 4. Air blank completed.
(X) 5. Calibration check completed. Test results 0. _____ AC.
(X) 6. Air blank completed.
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] ISS# 33629

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

03/10/2013

15:23:58

Standard Lot#: 15712100A1

Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014

YAVAPAI CSO

Operator: R. POOLE #9014

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	15:24:35
0.100 Cal Check	0.099	15:24:56
Air Blank	0.000	15:25:26

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST RT POOLE AGENCY YAVAPAI COUNTY S.O
DATE 3/10/13 TIME 1536
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
(X) 2. Date check.

OPERATIONAL TESTS


- (X) 1. Alcohol-free subject test result 0. 099 AC.
(X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES X NO

COMMENTS

400 PSI

SIGNATURE

 POOLE ESS QAS# 33629

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
03/10/2013

15:36:02

Standard Lot#: 15712100A1
Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014
YAVAPAI CSO

Operator: R. POOLE #9014
YAVAPAI CSO

Subject: SQAP,TEST,I
DOB: 01/01/1978
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:37:17
Diagnostic Test	Pass	15:37:51
Air Blank	0.000	15:38:19
0.100 Cal Check	0.099	15:38:39
Air Blank	0.000	15:39:09
Subject Test	0.000	15:39:29
Air Blank	ABT*	15:39:44
Air Blank	IPS**	15:40:06

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
03/10/2013

15:41:53

Standard Lot#: 15712100A1
Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014
YAVAPAI CSO

Operator: R. POOLE #9014
YAVAPAI CSO

Subject: SQAP,TEST,I
DOB: 01/01/1978
Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:43:04
Diagnostic Test	Pass	15:43:38
Air Blank	0.000	15:44:05
0.100 Cal Check	0.098	15:44:26
Air Blank	0.000	15:44:56
Subject Test	0.308*	15:48:03
Air Blank	RFI**	15:48:10
Air Blank	0.000	15:48:42

*Deficient Sample

**RFI Detect

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RT POOLE AGENCY YAVAPAI COUNTY S.O
DATE 3/10/13 TIME 1550
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(X) 4. Air blank completed.
(X) 5. Calibration check completed. Test results 0. 099 AC.
(X) 6. Air blank completed.
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  POOLE E39 QAS# 33629

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

03/10/2013

15:50:32

Standard Lot#: 15712100A1

Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014

YAVAPAI CSO

Operator: R. POOLE #9014

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	15:51:03
0.100 Cal Check	0.097	15:51:24
Air Blank	0.000	15:51:54

Preventative Maintenance
Performed -- Timer Reset