

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA-SPECIALIST RT FOOLE AGENCY YAVAPAI COUNTY SHERIFF
DATE 11/15/13 TIME 1638
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE TRAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(X) 4. Air blank completed.
(X) 5. Calibration check completed. Test results 0. 98 AC.
(X) 6. Air blank completed.
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] FOOLE PAS # 33629

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
11/15/2013 16:38:23

Standard Lot#: 15013100A2
Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014
YAVAPAI CSO

Operator: R. POOLE #9014
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	16:38:54
0.100 Cal Check	0.098	16:39:15
Air Blank	0.000	16:39:45

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST RT POLE AGENCY YAVAPAI COUNTY SHERIFF
DATE 11/15/13 TIME 1626 + 1632
INTOXILYZER SERIAL # 80 001222 LOCATION CAMP VERDE JAIL

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
(X) 2. Date check.

OPERATIONAL TESTS


- (X) 1. Alcohol-free subject test result 0. 100 AC.
(X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 99 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

STANDARD PSI AT 825

SIGNATURE

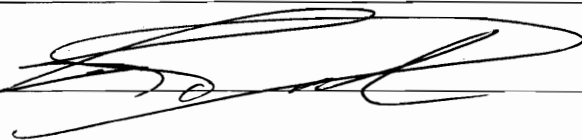

Pole QAS # 33629

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
11/15/2013

16:24:55

Standard Lot#: 15013100A2
Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014
YAVAPAI CSO

Operator: R. POOLE #9014
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1990
Sex: M Weight: 250

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:26:44
Diagnostic Test	Pass	16:27:18
Air Blank	0.000	16:27:46
0.100 Cal Check	0.099	16:28:06
Air Blank	0.000	16:28:36
Subject Test	0.000	16:28:58
Air Blank	ABT*	16:29:06
Air Blank	IPS**	16:29:14

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO-VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
11/15/2013

16:31:14

Standard Lot#: 15013100A2
Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014
YAVAPAI CSO

Operator: R. POOLE #9014
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1990
Sex: M Weight: 250

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	16:32:14
Diagnostic Test	Pass	16:32:48
Air Blank	0.000	16:33:15
0.100 Cal Check	0.098	16:33:36
Air Blank	0.000	16:34:06
Subject Test	DEF*	16:37:13
Air Blank	IPS**	16:37:20
Air Blank	RFI***	16:37:28

*Deficient Sample
**Improper Sample
***RFI Detect

Not a Successfully Completed Test Sequence


EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA-SPECIALIST R. T. POOLE AGENCY YAVAPAI COUNTY SHERIFF
DATE 11/15/13 TIME 1641
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. _____ AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

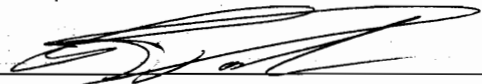SIGNATURE  POOLE PAS # 33629

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

11/15/2013

16:40:47

Standard Lot#: 15013100A2

Last Changed By: R. POOLE #9014

QAS: R. POOLE #9014

YAVAPAI CSO

Operator: R. POOLE #9014

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 16:41:18
0.100 Cal Check 0.099 16:41:39
Air Blank 0.000 16:42:09

Preventative Maintenance
Performed -- Timer Reset