

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.
DATE 11/30/2013 TIME 2235
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO - VERDE JAIL

- (S) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution of 100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
- (S) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (S) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (M) 4. Air blank completed.
- (S) 5. Calibration check completed. Test results 0. 092 AC.
- (S) 6. Air blank completed.
- (S) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE L. Moody 2486

725 psi

INTOXILYZER 8000

Location: YCSO-VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

11/30/2013

22:35:41

Standard Lot#: 15013100A2

Last Changed By: R. POOLE #9014

QAS: L. MOODY #7486

YAVAPAI CSO

Operator: L. MOODY #7486

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	22:37:31
0.100 Cal Check	0.092	22:37:51
Air Blank	0.000	22:38:21

Preventative Maintenance
Performed -- Timer Reset