

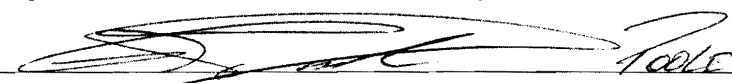
EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RT POOLE AGENCY YAVAPAI COUNTY S.O
DATE 7/16/14 TIME 12:54
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START" - CONTACT QAS
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 098 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  POOLE ESS QAS #33629

DPS Form Exh G-2 (Rev 05-01)

250 PSI

INTOXILYZER 8000

Location:

Serial Number: 80-001222

Core Version: 8105.48

07/16/2014

12:53:36

Standard Lot#: 15013100A2

Last Changed By: T. QAS #1234

QAS: R. POOLE #9014

YAVAPAI CSO

Operator: R. POOLE #9014

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	12:54:45
0.100 Cal Check	0.098	12:55:06
Air Blank	0.000	12:55:35

Preventative Maintenance
Performed -- Timer Reset