

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)


ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RT POOLE AGENCY YAVAPAI COUNTY S.O  
DATE 2/5/15 TIME 2019  
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START" CONTACT GAS
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. \_\_\_\_\_ AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

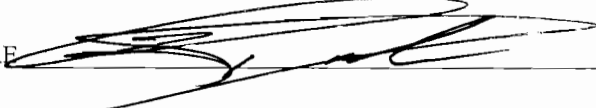

POOLE GAS# 33629

DPS Form Exh G-2 (Rev 05-01)

300 PSI



EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST RT POOLE AGENCY YAVAPAI COUNTY S.O  
DATE 2/5/15 TIME 2022  
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS


1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

300 PSI

SIGNATURE

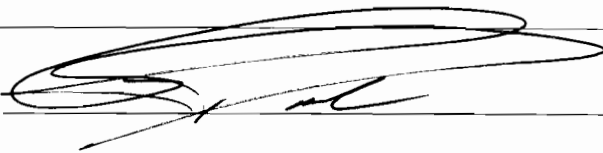

POOLE QAS # 33629

INTOXILYZER 8000

Location:

Serial Number: 80-001222

Core Version: 8105.48

02/05/2015

20:22:23

---

Standard Lot#: 33913100A3

Last Changed By: R. POOLE #9014

---

QAS: R. POOLE #9014

YAVAPAI CSO

---

Operator: R. POOLE #9014

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M Weight: 250

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	20:23:34
Diagnostic Test	Pass	20:24:09
Air Blank	0.000	20:24:37
0.100 Cal Check	0.099	20:24:57
Air Blank	0.000	20:25:27
Subject Test	0.000	20:25:51
Air Blank	ABT*	20:26:00
Air Blank	IPS**	20:26:08

\*Sequence Aborted

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location:

Serial Number: 80-001222

Core Version: 8105.48

02/05/2015

20:28:04

---

Standard Lot#: 33913100A3

Last Changed By: R. POOLE

#9014

---

QAS: R. POOLE

#9014

YAVAPAI CSO

---

Operator: R. POOLE

#9014

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 250

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	20:29:02
Diagnostic Test	Pass	20:29:37
Air Blank	0.000	20:30:05
0.100 Cal Check	0.098	20:30:26
Air Blank	0.000	20:30:55
Subject Test	DEF*	20:34:03
Air Blank	0.000	20:34:30
Air Blank	IPS**	20:34:37
Air Blank	RFI***	20:34:46

\*Deficient Sample

\*\*Improper Sample

\*\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RT POOLE AGENCY YAVAPAI COUNTY S.O  
DATE 2/5/15 TIME 2035  
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution \_\_\_\_\_ AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature] POOLE QAS# 33629

300 PSE

INTOXILYZER 8000

Location:

Serial Number: 80-001222

Core Version: 8105.48

02/05/2015

20:35:43

-----  
Standard Lot#: 33913100A3

Last Changed By: R. POOLE #9014  
-----

QAS: R. POOLE #9014

YAVAPAI CSO  
-----

Operator: R. POOLE #9014

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	20:36:14
0.100 Cal Check	0.098	20:36:35
Air Blank	0.000	20:37:05

Preventative Maintenance  
Performed -- Timer Reset