

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.
DATE 12/22/2015 TIME 1747
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO - CAMP VERDE JAIL

- (L) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (L) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(L) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(L) 4. Air blank completed.
(L) 5. Calibration check completed. Test results 0. 098 AC.
(L) 6. Air blank completed.
(L) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE LOGAN MOODY

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
12/22/2015 17:47:16

Standard Lot#: 26614100A6
Last Changed By: R. POOLE #9014

QAS: L. MOODY #7486
YAVAPAI CSO

Operator: L. MOODY #7486
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	17:47:45
0.100 Cal Check	0.098	17:48:06
Air Blank	0.000	17:48:36

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. SO.
DATE 12/22/2015 TIME 1748
INTOXILYZER SERIAL # 80-001222 LOCATION GUSD - CAMP VERDE JAIL

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

550 psi

SIGNATURE

Logan Moody 7416

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

12/22/2015

17:48:48

Standard Lot#: 26614100A6

Last Changed By: R. POOLE #9014

QAS: L. MOODY #7486

YAVAPAI CSO

Operator: L. MOODY #7486

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/2000

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:49:34
Diagnostic Test	Pass	17:50:09
Air Blank	0.000	17:50:37
0.100 Cal Check	0.098	17:50:57
Air Blank	0.000	17:51:27
Subject Test	0.000	17:51:45
Air Blank	ABT*	17:51:56
Air Blank	RFI**	17:52:03

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

12/22/2015

17:52:07

Standard Lot#: 26614100A6

Last Changed By: R. POOLE

#9014

QAS: L. MOODY

#7486

YAVAPAI CSO

Operator: L. MOODY

#7486

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/2000

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:52:52
Diagnostic Test	Pass	17:53:27
Air Blank	0.000	17:53:55
0.100 Cal Check	RFI*	17:54:15
Air Blank	RFI*	17:54:23

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

12/22/2015

17:54:51

Standard Lot#: 26614100A6

Last Changed By: R. POOLE

#9014

QAS: L. MOODY

#7486

YAVAPAI CSO

Operator: L. MOODY

#7486

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/2000

Sex: M

Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:55:39
Diagnostic Test	Pass	17:56:14
Air Blank	0.000	17:56:41
0.100 Cal Check	0.099	17:57:02
Air Blank	0.000	17:57:31
Subject Test	RFI*	17:59:16
Air Blank	0.000	17:59:49

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

12/22/2015

18:00:11

Standard Lot#: 26614100A6

Last Changed By: R. POOLE #9014

QAS: L. MOODY #7486

YAVAPAI CSO

Operator: L. MOODY #7486

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/2000

Sex: M Weight: 200

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:00:56
Diagnostic Test	Pass	18:01:31
Air Blank	0.000	18:01:59
0.100 Cal Check	0.098	18:02:20
Air Blank	0.000	18:02:49
Subject Test	0.077*	18:05:57
Air Blank	0.000	18:06:28
Air Blank	0.000	18:06:56
Subject Test	XXX**	18:07:13
Air Blank	IPS***	18:07:20

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***Improper Sample

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.
DATE 12/22/2015 TIME 1807
INTOXILYZER SERIAL # 80-001222 LOCATION YCSO - CAMP VERDE JAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 1.00 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 098 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

12/22/2015

18:07:36

Standard Lot#: 26614100A6

Last Changed By: R. POOLE #9014

QAS: L. MOODY #7486

YAVAPAI CSO

Operator: L. MOODY #7486

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	18:08:05
0.100 Cal Check	0.098	18:08:26
Air Blank	0.000	18:08:56

Preventative Maintenance
Performed -- Timer Reset