

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SAAN CHUPP AGENCY YCSO  
DATE 5/22/16 TIME 1947  
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE SAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 099 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  539/9808

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

05/22/2016

19:47:27

-----  
Standard Lot#: 26614100A6

Last Changed By: S. CHUPP #9808  
-----

QAS: S. CHUPP #9808

YAVAPAI CSO  
-----

Operator: S. CHUPP #9808

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	19:48:00
0.100 Cal Check	0.099	19:48:21
Air Blank	0.000	19:48:50

Preventative Maintenance  
Performed -- Timer Reset