

DR 17-002829

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
01/23/2017 23:28:51

Standard Lot#: 22315100A1
Last Changed By: E. BOONE #5823

QAS: S. CHUPP #9808
YAVAPAI CSO

Operator: S. CHUPP #9808
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:29:24
0.100 Cal Check	0.097	23:29:45
Air Blank	0.000	23:30:14

Preventative Maintenance
Performed -- Timer Reset

DR 17-002829

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SEAN CHUPP AGENCY YCSO
DATE 01/23/17 TIME 23:20
INTOXILYZER SERIAL # 80-001222 LOCATION CV JAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 097 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

DR 17-002829

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
01/23/2017 23:28:51

Standard Lot#: 22315100A1
Last Changed By: E. BOONE #5823

QAS: S. CHUPP #9808
YAVAPAI CSO

Operator: S. CHUPP #9808
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:29:24
0.100 Cal Check	0.097	23:29:45
Air Blank	0.000	23:30:14

Preventative Maintenance
Performed -- Timer Reset