

17-006412

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SEAN CHUPP AGENCY YCSO  
DATE 2/20/17 TIME 1951  
INTOXILYZER SERIAL # 80-001222 LOCATION CV SAIL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 099 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  100 PS 1 \*

17-006712

INTOXILYZER 8000  
Location: YCSO - VERDE JAIL  
Serial Number: 80-001222  
Core Version: 8105.48  
02/20/2017 19:51:35

-----  
Standard Lot#: 22315100A1  
Last Changed By: E. BOONE #5823  
-----

QAS: S. CHUPP #9808  
YAVAPAI CSO  
-----

Operator: S. CHUPP #9808  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	19:52:08
0.100 Cal Check	0.099	19:52:29
Air Blank	0.000	19:52:59

Preventative Maintenance  
Performed -- Timer Reset



INTOXILYZER 8000  
Location: YCSO - VERDE JAIL  
Serial Number: 80-001222  
Core Version: 8105.48  
03/20/2017 15:41:17

-----  
Standard Lot#: 22315100A1  
Last Changed By: E. BOONE #5823  
-----

QAS: S. CHUPP #9808  
YAVAPAI CSO  
-----

Operator: S. CHUPP #9808  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	15:41:47
0.100 Cal Check	0.097	15:42:08
Air Blank	0.000	15:42:38

Preventative Maintenance  
Performed -- Timer Reset

17-009822

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)  
ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SEAN CHUPP AGENCY YCSO  
DATE 03/20/19 TIME 1520  
INTOXILYZER SERIAL # 80-001222 LOCATION CV JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
~~Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(X) 4. Air blank completed.  
(X) 5. Calibration check completed. Test results 0. 098 AC.  
(X) 6. Air blank completed.  
(X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000  
Location: YCSO - VERDE JAIL  
Serial Number: 80-001222  
Core Version: 8105.48  
03/20/2017 15:20:07

-----  
Standard Lot#: 22315100A1  
Last Changed By: E. BOONE #5823  
-----

QAS: S. CHUPP #9808  
YAVAPAI CSO  
-----

Operator: S. CHUPP #9808  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	15:20:41
0.100 Cal Check	0.098	15:21:02
Air Blank	0.000	15:21:31

Preventative Maintenance  
Performed -- Timer Reset

17-009822

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST SAAN CHUPP AGENCY YCSO  
DATE 03/20/19 TIME 1522  
INTOXILYZER SERIAL # 80-001222 LOCATION CV JAIL

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 090 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

ISO PSI

SIGNATURE



INTOXILYZER 8000  
Location: YCSO - VERDE JAIL  
Serial Number: 80-001222  
Core Version: 8105.48  
03/20/2017 15:22:41

---

Standard Lot#: 22315100A1  
Last Changed By: E. BOONE #5823

---

QAS: S. CHUPP #9808  
YAVAPAI CSO

---

Operator: S. CHUPP #9808  
YAVAPAI CSO

---

Subject: SQAP,TEST,I  
DOB: 01/01/1990  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:23:52
Diagnostic Test	Pass	15:24:27
Air Blank	0.000	15:24:54
0.100 Cal Check	0.098	15:25:15
Air Blank	0.000	15:25:45
Subject Test	0.000	15:26:01
Air Blank	ABT*	15:26:08
Air Blank	RFI**	15:26:23

\*Sequence Aborted

\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO - VERDE JAIL  
Serial Number: 80-001222  
Core Version: 8105.48  
03/20/2017

15:27:16

---

Standard Lot#: 22315100A1  
Last Changed By: E. BOONE #5823

---

QAS: S. CHUPP #9808  
YAVAPAI CSO

---

Operator: S. CHUPP #9808  
YAVAPAI CSO

---

Subject: SQAP, TEST, I  
DOB: 01/01/1990  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	15:28:10
Air Blank	0.000	15:28:38

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO - VERDE JAIL  
Serial Number: 80-001222  
Core Version: 8105.48  
03/20/2017

15:28:58

---

Standard Lot#: 22315100A1  
Last Changed By: E. BOONE #5823

---

QAS: S. CHUPP #9808  
YAVAPAI CSO

---

Operator: S. CHUPP #9808  
YAVAPAI CSO

---

Subject: SQAP,TEST,I  
DOB: 01/01/1990  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:29:52
Diagnostic Test	Pass	15:30:27
Air Blank	0.000	15:30:54
0.100 Cal Check	0.097	15:31:15
Air Blank	0.000	15:31:45
Subject Test	0.000*	15:34:52
Air Blank	0.000	15:35:24
Air Blank	0.000	15:35:51
Subject Test	RGE**	15:36:08
Air Blank	0.000	15:36:44

\*Deficient Sample

\*\*Range Exceeded

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO - VERDE JAIL  
Serial Number: 80-001222  
Core Version: 8105.48  
03/20/2017 15:36:57

---

Standard Lot#: 22315100A1  
Last Changed By: E. BOONE #5823

---

QAS: S. CHUPP #9808  
YAVAPAI CSO

---

Operator: S. CHUPP #9808  
YAVAPAI CSO

---

Subject: SQAP,TEST,I  
DOB: 01/01/1990  
Sex: M Weight: 180

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:37:52
Diagnostic Test	Pass	15:38:27
Air Blank	0.000	15:38:54
0.100 Cal Check	0.097	15:39:15
Air Blank	0.000	15:39:44
Subject Test	XXX*	15:40:08
Air Blank	0.000	15:40:42

\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**